Verstärkungsgitter
Mesh Strengtheners
Grilles de renforcement
Refuerzos de Paladares
Reti di rinforzo

- Informationen zu Produkten finden Sie unter www.dentaurum.com
- For more information on our products, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- Descubra nuestros productos en www.dentaurum.com
- Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 05/16

Änderungen vorbehalten | Subject to modifications |
Sous réserve de modifications | Reservado el derecho
de modificación | Con riserva di apportare modifiche

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 7231/803-0 | Fax +49 7231/803-295
www.dentaurum.com | info@dentaurum.com

DE

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EN

Dear customer

Thank you for choosing a quality product from Dentaurum.
It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.
Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. Instructions for use

Dentaurum mesh strengtheners are used in reinforcing acrylic full and partial dentures. They are substantially different from the usual metallic or plastic mesh-type reinforcers hitherto available to the profession. Dentaurum mesh strengtheners are anatomically preformed plates having apertures, approximately 2.5 mm in diameter, that are closely spaced and which permit the mesh to be easily shaped. The apertures ensure that the acrylic resin layers above and below the mesh are strongly bonded to each other; thus the mesh will not disrupt the integrity of the denture base material. Available as stainless steel or gold-plated stainless steel in thicknesses of 0.4 mm (16 inch).

Instructions for use:

The partially preformed strengtheners are adjusted for each case. This can be done simply and quickly. For models of edentulous jaws the master model is covered with a single thickness of 1.0 mm modelling wax. This is then duplicated in Rapidur®. An opposing counter die can be formed using several layers of newspaper. Only in cases having deep palatal vaults is it advisable to construct a counter die of Rapidur® or some other firm material.

The mesh strengtheners, which are supplied in a softened condition, can be adapted to the duplicate model by applying positive pressure to the blank via the paper counterdie. An orthodox laboratory flask press can be used for this. By slowly increasing the pressure the mesh will be adapted to the contours of the duplicate model. The edge of the strengthener should be trimmed to extend just beyond the alveolar ridge crest. Dorsally, the strengthener should terminate 2.0 mm in front of the intended posterior border of the denture base.

When introducing the strengthener into the denture base, preliminary packing of the resin should take place with the previously used wax spacer in position. Subsequent to the first trial closure, the wax is removed and the remaining space filled with additional resin. Process the denture in the usual way. It is only necessary, during this, to ensure that the mesh strengthener is completely covered by the acrylic and that it is not exposed during any polishing or trimming of the denture.

3. Delivery program

Stainless steel	0.4 mm	10 pieces	REF 318-004-00
gold-plated	0.4 mm	10 pieces	REF 318-104-00

4. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

5. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of the symbols used on the label REF 989-313-00).

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