

Silfix



Technical Data Sheet

Intended use: Addition-vulcanising high precision embedding-silicone for full dentures

Manufacturer: Dreve Dentamid GmbH
Max-Planck-Straße 31
59423 Unna / Germany

Composition:

Pos	Chemical classification
1	venyl containing polydemethylsiloxane
2	silicic acid
3	filling materials
4	pigments

Colour: rose

Final hardness: 74 ± 2 Shore A

Type: type 3; low consistency

Mixing time: 30 s

Processing time: 2 min ± 1 min at 23°C

Deflasking time: 7 min ± 30 sec at 23°C

Dosage: Mix components A + B in a 1 : 1 ratio

Dreve Dentamid GmbH
Max-Planck-Str. 31
59423 Unna / Germany
Tel.: 02303 / 8807-0
Fax: 02303 / 8807-55
dentamid@dreve.de
www.dentamid.dreve.de