Essentia® Light-cured radiopaque universal composite restorative

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Direct restorative for class I, II, III, IV, V cavities.
- Direct restorative for wedge-shaped defects and root surface cavities.
- Direct restorative for veneers and diastema closure.

CONTRAINDICATIONS

- Pulpitis.
- Avoid use of this product in patients with known allergies to methacrylate monomers or methacrylate polymer.

DIRECTIONS FOR USE

1. Shade Selection

- Mask the tooth to be restored is clean and not dehydrated.
- Select first an enamel shade, looking to the incisal third of the (neighboring) tooth. Based on the observed value (grayness), choose between Light Enamel (white) or Dark Enamel (grey).
- Continue with the selection of the dentine shade, looking to the cervical part of the (neighboring) tooth. Based on the observed value (chroma (intensity of the color), choose between Light Dentin (low chroma), Medium Dentin and Dark Dentin (high chroma).
- In small or posterior restorations, the Universal Shade will match most of the cases.
- To mask discolorations, place a thin layer of the Masking Liner over the discoloration.

2. Filling Preparation

- Prepare cavity using standard techniques. Dry by gently blowing with oil free air.
- Note: For pulp capping, use calcium hydroxide.

3. Bonding Treatment

- For bonding of Essentia to enamel and / or dentin, use a light-cured bonding system such as G-Premio BOND, G-enial® Bond or G-BOND®.
- Follow the manufacturer's instructions.

4. Placement of Essentia

- Masking Liner**
 - Hold the Masking Liner against and remove the cap by turning counterclockwise. Promptly and securely attach the dispensing tip to the syringe by pulling the cap off.
 - For the extrusion of the material, gently trial-push the syringe plunger outside the mouth to make sure that the dispensing tip is securely attached to the syringe.
 - Note: If there is air inside the dispensing tip, bubbles may be formed at the time of injection.
 - Place the dispensing tip as close as possible to the cavity, and slowly push the plunger to inject material into the cavity. Alternatively, dispense material onto a mixing pad and transfer the cavity using a suitable instrument.
 - After use, immediately remove and dispose of the dispensing tip and tightly close the syringe with the cap.

Dentin, Enamel, Universal shades and Opalescent modifier

- Dispensing from a syringe
 - Open syringe cap and dispense material into a mixing pad. Place the material into the cavity using a suitable placement instrument. After dispensing, screw plunger anticlockwise by a half turn to release residual paste inside the syringe. Close cap immediately after use.

Red-Brown, Black, White

- Dispensing from a syringe
 - Open syringe cap and dispense material into a mixing pad. Apply the material on the Dentin shade or Enamel shade using a brush or a suitable placement instrument.

Note: **Modificator stains** are inner stain materials. They should always be covered with Enamel shades or Opalescent modifier after light curing.

- The Masking Liner can be used to mask cavity bottom discolorations.
- The Universal shade can be used to achieve aesthetic posterior restorations in a single layer.
- The Dentin and Enamel shades should be used following a dual-layering concept. Refer to ClinHint for additional details.
- A the Composite can be stored in the fridge in order to obtain a firmer consistency. Material may be hard to extrude immediately after coming from cold storage. Prior to use, let it stand for a few minutes at room temperature.
- Layering with a brush wetted with a modeling liquid can ease the shaping of the composite.
- After use, immediately remove the dispensing tip. Ambient light can shorten the manipulation time.

Clinical Hints

a. In the case of small cavities

Restore using a one shade technique. In most cases, a single shade of the Universal shade alone will be sufficient. In cases where a higher degree of translucency is needed, a combination of Dentin and Enamel shades can be selected.

b. In the case of large and/or deep cavities

In most cases a multi-shade layering technique will give the best aesthetic results. To mask discolored dentin, use the Masking Liner and dentin shade, and then cover it with an Enamel shade.

Clinical Hint 4.1 - EXAMPLES OF CLINICAL APPLICATIONS

Blackch Junior	Young	Adult	Senior	Posterior Mono	Posterior Duo
Dentin & Dark Enamel	Medium Dentin & Light Enamel	Medium Dentin & Dark Enamel	Dentin & Dark Enamel	Universal Enamel	Dentin & Dark Enamel

Clinical Hint 4.2 - MAIN SHADE COMBINATIONS

Blackch Junior	Young	Adult	Senior	Posterior Mono	Posterior Duo
Dentin & Dark Enamel	Medium Dentin & Light Enamel	Medium Dentin & Dark Enamel	Dentin & Dark Enamel	Universal Enamel	Dentin & Dark Enamel

c. Use of the modifier syringes

- The Opalescent modifier can be used to add opalescence to the incisal area, using a small pluggins into the enamel layer.

- Red-Brown, Black and White modifiers are liquid stains used to characterize the restoration.

5. Contouring before light-curing

Contour using standard techniques.

6. Light-curing

Light cure Essentia using a light curing unit. Keep the light source as close as possible to the surface. Due to the following chart for Irradiation Time and Effective Depth of Cure

Irradiation Time and Effective Depth of Cure

Dentin Shades	20 sec. Irradiation (Average LED 7000mW/cm²) 10 sec. (LED 14000 wattage) (vs. 12000mW/cm²)	LD	MD	DD	U	ML	OM	RBM	BM	WM
Enamel & Universal Shades	LD	MD	DD	U	ML	OM	RBM	BM	WM	
Masking Liner	LD	MD	DD	U	ML	OM	RBM	BM	WM	

Note:

- Material should be placed and light-cured in layers.
- For minimum layer thickness, please consult the above table.
- A lower light intensity may cause insufficient curing or discoloration of the material.

Finishing and Polishing

Finish and polish using diamond burs, polishing points and discs. To obtain a high gloss, polishing pastes can be used.

SHES

- LD: Light Dentin
- MD: Medium Dentin
- DD: Dark Dentin
- LE: Light Enamel
- DE: Dark Enamel
- U: Universal
- ML: Masking Liner
- OM: Opalescent Modifier
- RB: Red-Brown
- BM: Black Modifier
- WM: White Modifier

STORAGE

For best and optimal performance, store in a cool and dark place (4 - 25°C / 39.2 - 77°F).

PACKAGES

1. Syringes

- Essentia Starter Kit Syringe**
 - 7 syringes (one per each shade: LD, MD, DD, LE, DE, U; ML)
 - 20 Dispensing Tips III Plastic + 1 light-protective cover

2. Essentia Modifier Kit

- 4 Syringes: OM, RBM, BM, WM
- 4 Syringes: Springe Refills

LD, MD, DD, LE, DE, U; OM, RBM, BM, WM: available in refill of 1 syringe each

ML: refill of 1 syringe including 20 Dispensing Tips III Plastic

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