# Master all your challenges

*initial*<sup>™</sup>

FIRING FOAM

- EN Technical Manual
- DE Technisches Handbuch
- FR Manuel Technique
- IT Manuale tecnico
- ES Manual técnico
- PL Instrukcja techniczna

OCTOBER 2018

# GC Initial FIRING FOAM

## Special refractory support material for ceramic firing

The GC Initial FIRING FOAM for all GC Initial ceramic systems ensures a stable fixing of objects onto the ceramic firing trays.

### INDICATIONS


- Inlays, onlays, crowns, veneers, bridges

### BENEFITS

- Ease of application due to its special creamy consistency
- Guaranteed dimensional stability during the firing process
- Perfect distribution of heat onto the object
- Easy to remove after firing

### DIRECTIONS FOR USE

#### APPLICATION

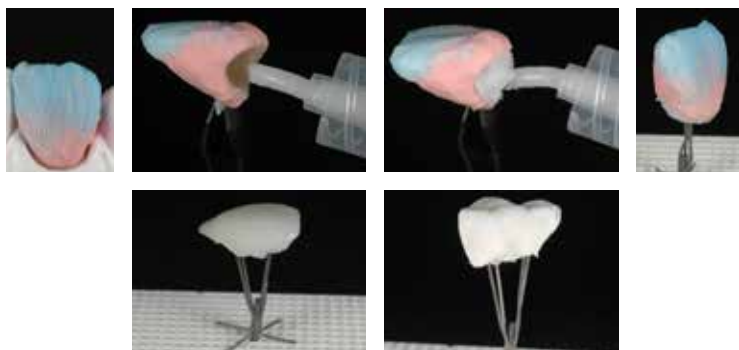- Apply FIRING FOAM directly from the syringe into the crown or onto the pressed ceramic object
- Carefully place crown or pressed ceramic object on the pin of the ceramic firing tray

#### REMOVAL

- After firing, quickly remove the solidified FIRING FOAM under running water with an instrument
- If necessary, blast off with non-abrasive glass beads under a maximum pressure of 2 bar

#### REMARK

Avoid contact of the FIRING FOAM to the ceramic. Use only as much FIRING FOAM as is needed. When using low fusing ceramic types (e.g. GC Initial LF / GC Initial LiSi) increase the drying time up to 8 minutes.

### STORAGE

Recommended for optimal performance, close the syringe to ensure an airtight seal. Store in a cool and dry place.

### PACKAGES

1x GC Initial FIRING FOAM, 10g

### CAUTION

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas: <http://www.gcamerica.com>. They can also be obtained from your supplier.

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- Inlays, Onlays, Kronen, Veneere, Brücken

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GC AMERICA INC.  
3737 West 127th Street, USA-Alsip, IL. 60803  
Tel. +1.708.597.0900, Fax. +1.708.371.5103  
sales@gcamerica.com, www.gcamerica.com

GC Germany GmbH  
Seifgrundstr. 2, D-61348 Bad Homburg  
Tel. +49.61.72.99.59.60, Fax. +49.61.72.99.59.66  
info.germany@gc.dental, www.germany.gceurope.com

GC ITALIA S.r.l.  
Via Calabria 1, I-20098 San Giuliano Milanese  
Tel. +39.02.98.28.20.68, Fax. +39.02.98.28.21.00  
info.italy@gc.dental, www.italy.gceurope.com

GC UNITED KINGDOM Ltd.  
16-23 Coopers Court, Coopers Court,  
Newport Pagnell, UK-Bucks. MK16 8JS  
Tel. +44.1908.218.999, Fax. +44.1908.218.900  
info.uk@gc.dental, www.uk.gceurope.com

GC FRANCE s.a.s.  
8 rue Benjamin Franklin, F-94370 Sucy en Brie Cedex  
Tel. +33.1.49.80.37.91, Fax. +33.1.45.76.32.68  
info.france@gc.dental, www.france.gceurope.com

GC IBÉRICA - Dental Products, S.L.  
Edificio Codesa 2  
Playa de las Américas, 2, 1º, Of. 4,  
ES-28290 Las Rozas, Madrid  
Tel. +34.916.364.340, Fax. +34.916.364.341  
comercial.spain@gc.dental, www.spain.gceurope.com

GC AUSTRIA GmbH  
Tallak 124, A-8103 Rein bei Graz  
Tel. +43.312.45.40.20, Fax. +43.312.45.40.20.40  
info.austria@gc.dental, www.austria.gceurope.com

GC BENELUX B.V.  
Edisonbaan 12, NL-3439 MN Nieuwegein  
Tel. +31.30.630.85.00, Fax. +31.30.605.59.86  
info.benelux@gc.dental, www.benelux.gceurope.com

GC EUROPE N.V. - East European Office  
Siget 19b, HR-10020 Zagreb  
Tel. +385.1.46.78.474, Fax. +385.1.46.78.473  
info.eeo@gc.dental, www.eeo.gceurope.com

GC NORDIC AB - Finnish Branch  
Bertel Jungin aukio 5 (6. kerros), FIN-02600 Espoo  
Tel. +358 40 9000 757  
info.finland@gc.dental, www.finland.gceurope.com

GC NORDIC AB - Danish Branch  
Scandinavian Trade Building  
Gydevang 39-41, DK-3450 Allerød  
Tel: +46 768 544 350  
info.denmark@gc.dental, http://nordic.gceurope.com

GC NORDIC AB  
Strandvägen 54 - S-193 30 Sigtuna  
Tel: +46 768 544 350  
info.nordic@gc.dental, http://nordic.gceurope.com

GC AUSTRIA GmbH - Swiss Office  
Bergstrasse 31, CH-8890 Flums  
Tel. +41.81.734.02.70, Fax. +41.81.734.02.71  
info.switzerland@gc.dental, www.switzerland.gceurope.com

GC AUSTRALASIA DENTAL PTY LTD  
1753 Botany Rd, Banksmeadow  
Sydney, NSW 2019, Australia  
Tel: +61.2.9301.8200, Fax: +61.2.9316.4196  
www.gcaustralasia.com

# GC Initial™ FIRING FOAM

- EN SPECIAL REFRACTORY SUPPORT MATERIAL FOR CERAMIC FIRING
- DE FEUERFESTES MATERIAL FÜR DEN KERAMIK-BRAND
- FR MATERIAU DE SUPPORT REFRACTAIRE POUR CUISSON DE LA CERAMIQUE
- IT MATERIALE SPECIALE REFATTARIO DI SUPPORTO PER LA COTTURA DELLA CERAMICA
- ES MATERIAL DE SOPORTE REFRACTARIO ESPECIAL PARA LA COCCIÓN DE CERÁMICA
- NL VUURVAST MATERIAAL VOOR HET OPBAKKEN VAN KERAMIEK
- PT MATERIAL DE SUPORTE REFRACTÁRIO ESPECIAL PARA COCÇÃO DE CERÂMICA
- DA SPECIEL BRÆNDGIPS MED YDERLIGERE SUPPORT VED PORCELÆNBÆRÆDNING
- SV SPECIELLT BRÄNNGIPS FÖR YTTERLIGARE STÖD VID PORSLINSBRÄNNING
- EL ΕΙΔΙΚΟ ΥΛΙΚΟ ΥΠΟΣΤΗΡΙΞΗΣ ΤΥΠΟΧΩΜΑΤΩΣΗΣ ΓΙΑ ΟΡΤΗΣ ΚΕΡΑΜΙΚΩΝ
- PL SPECJALNY OGNIOTRWAŁY MATERIAŁ POMOCNICZY PRZY WYPALANIU CERAMIKI
- RU СРЕДНУЛЫБЫЙ ОГНЕУПОРНЫЙ БСЛОМОУАТЕЛЫБЫЙ МАТЕРИАЛ ДЛН ОВКНН КЕРАМНКН
- TR SERAMİK PİŞİRME İÇİN ÖZEL REFRAKTORY DESTEK MATERİYALI

QTY: 1 x 10g

GC EUROPE N.V.  
INTERLEUVENLAAN 33  
LEUVEN, BELGIUM

Manufacturer  
EU

GC Europe N.V.  
Interleuvenlaan 33  
B-3001 Leuven

DISTRIBUTED by

GC EUROPE N.V.  
B-3001 LEUVEN, BELGIUM  
SINGAPORE 528729

GC AMERICA INC.  
ALSP, IL 60803 U.S.A.  
www.gcamerica.com

DISTRIBUTED by  
AND RESPONSIBLE MANUFACTURER  
IN CANADA

Consult Instructions  
for Use

Recommended for optimal  
performance, store in  
a cool and dry place.

MADE IN AUSTRIA  
1029142 - Y0903