Prior to use, carefully read the instructions for use.

## METAL PRIMER Z

PRIMER FOR BONDING RESIN TO METAL AND ZIRCONIA

METAL PRIMER Z is a priming agent designed to treat the adhering surface of metal, zirconia and prostheses to promote a strong bond to resin materials such as resin-based cements, laboratory resins, direct composites and acrylics.

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**

Bonding of acrylic and composite resins to metal and zirconia prostheses as set out below in the DIRECTIONS FOR USE.

**CONTRAINDICATIONS**

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or ethanol.

**RECOMMENDED APPLICATIONS**

- Application of opaque resin to metal for composite resin faced metal crowns
- Cementation of inlays, crowns and bridges with resin cement
- Bonding of denture base to magnetic attachment
- Bonding of denture base resin to metal base, clasp or attachment
- Repair of denture with metal base, clasp or attachment

Note:

After flushing out the wax, clean the bonding surface using a steam cleaner. Remove wax completely using solvent before applying METAL PRIMER Z.
- Bonding resin to metal orthodontic appliances

**DIRECTIONS FOR USE**

EXTRA-ORAL application in all cases

- Sandblast metal surfaces with alumina (aluminium oxide), and blow clean with air gun. Avoid contamination with oil, separator or moisture.
- Dispense 2-3 drops of METAL PRIMER Z into a clean dispensing dish. Using a brush, apply a thin layer to the bonding surface. Allow to dry.
- Immediately apply a resin material to the treated surface according to the instructions of the manufacturer.

INTRA-ORAL application in case of repair of prostheses.

- Place rubber dam.

Note:

Be sure to use rubber dam and avoid direct contact with oral tissue.
- Roughen the metal surface with a sandblaster or a coarse diamond bur. Avoid contamination with oil, separator or moisture.
- Dispense 2-3 drops of METAL PRIMER Z into a clean dispensing dish. Using a brush, apply a thin layer to the bonding surface. Allow to dry. A second coat may be applied if necessary.
- Immediately apply a resin material to the treated surface according to the instructions of the manufacturer.

**STORAGE**

Recommended for optimal performance, store at room temperature (4-25°C) (39,2-77,0°F). Shelf life: 2 years from date of manufacture.

**PACKAGE**

METAL PRIMER Z 5mL (1)

**CAUTION**

- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- METAL PRIMER Z is highly flammable liquid and vapour. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
- Take care to avoid ingestion of the material.
- In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Flush immediately after cementing.
- In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Replace the cap immediately after use. If dew condensation is present on the bottle of METAL PRIMER Z, immediately wipe off.
- Do not mix with any other primers.
- Store in a well ventilated place.
- Dispose of all waste according to local regulations.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: http://www.gceurope.com or for The Americas http://www.gcamerica.com They can also be obtained from your supplier.

**CLEANING AND DISINFECTING:** MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised: 06/2017

**CE 0086**

**'GC'**

MANUFACTURED by **GC DENTAL PRODUCTS CORP.** 2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan
**RESPONSIBLE MANUFACTURER IN CANADA:** **GC AMERICA INC.** 3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED by **GC CORPORATION** 76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU : GC EUROPE N.V.** Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

**GC AMERICA INC.** 3737 West 127th Street, Alsip, IL 60803 U.S.A. TEL: +1-708-597-0900 www.gcamerica.com

**GC ASIA DENTAL PTE. LTD.** 11 Tampines Concourse, #03-05, Singapore 528729 TEL: +65 6546 7588

**GC AUSTRALASIA DENTAL PTY. LTD.** 1753 Botany Rd, Bankmeadow, NSW 2019, Australia TEL: +61 2 9301 8200

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