# OPTOSIL® Liquid

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 Instructions for use

 Mode d'emploi

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 **Manufacturer:**  
Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau (Germany)  
Made in Romania  
Importado e Distribuído no Brasil por  
Kulzer South América Ltda.  
CNPJ 48.708.010/0001-02  
Rua Cenno Sbrighi, 27 – cj. 42  
São Paulo – SP – CEP 05036-010  
sac@kulzer-dental.com  
Resp. Técnica: Dra. Regiane Marton  
CRO 70.705  
N° ANVISA: vide embalagem

66056473/12

## OPTOSIL® liquid Elastomeric Isolating Material for Dental Technology

**Use:**  
Visually inspect for damage before first use. Damaged products may not be used.  
OPTOSIL® liquid is an isolating material for acrylic resin dentures.  
OPTOSIL® liquid prevent damage on the acrylic resin caused by water vapour.  
OPTOSIL® liquid gives a smooth, glossy finish.  
OPTOSIL® liquid is also suitable for reproducing gingival margins on precision models.  
Consistency: Free flowing  
Activator: Activator Universal Plus (OPTOSIL®/XANTOPREN®), Liquid  
Dosage: – for full upper and lower dentures (28 teeth)  
Fill the measuring cup to the upper ring with OPTOSIL® liquid (approx. 14 g) and add 14 drops (approx. 0.31 g) Activator Universal Plus, Liquid  
– for full upper or lower dentures (14 teeth)  
Fill the measuring cup to the lower ring with OPTOSIL® liquid (approx. 9 g) and add (approx. 0.20 g) Activator Universal Plus, Liquid

The stated dosage should be observed. Close carefully after use.

Mixing time: 30 seconds  
Working time\*: approx. 4 minutes  
Resume working after\*: approx. 10 minutes

\* from start of mixing / at room temperature  
Store OPTOSIL® liquid and Activator Universal Plus at room temperature (max. 25°C (77°F)).

**Sample application** – isolation of full upper or lower dentures (14 teeth)  
– Ensure OPTOSIL® liquid is well mixed in the bottle.  
– Fill measuring cup to lower ring with OPTOSIL® liquid.  
– Add Activator Universal Plus, Liquid (9 drops) and mix for 30 seconds.  
– Apply OPTOSIL® liquid thinly with a brush or spatula to the areas to be isolated allowing it to flow into the interdental spaces.  
– Do not coat occlusal surfaces and cutting (incisal) edges with OPTOSIL® liquid.  
– Sprinkle the adhesive granules onto the OPTOSIL® liquid while the latter is still plastic. Do not press them into the material.  
– After investment, boil out the wax in the usual way. The adhesive granules ensure secure bonding to the plaster.  
– Process the resin in the usual way.  
– On deflasking the cured denture, the film of OPTOSIL® liquid is easily removed from the teeth and the denture resin, ensuring that no plaster remains stuck to either.  
– The film of OPTOSIL® liquid renders finishing or interdental and gingival regions unnecessary.

**Warnings and precautions Activator Universal Plus:**  
Inflammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In case of contact with eyes, rinse with water and seek medical attention. Avoid skin contact. Wear protective gloves/protective clothing/eye protection/face protection (EN 374, category III, e.g. Dermatrill). Do not swallow or ingest. Hypersensitive persons may experience an allergic reaction to the product. Not suitable for use by pregnant or nursing women. Only to be used by qualified personnel in a dental laboratory. Not for use on humans. For disposal please follow the instructions on the material safety data sheet and national regulations. Safety data sheets and more information are available at our website [www.kulzer-dental.com](http://www.kulzer-dental.com)

**Presentation:**  
Combipack containing: 560 ml OPTOSIL® liquid – 25 ml Activator Universal Plus (OPTOSIL®/XANTOPREN®), Liquid – 500 g Adhesive granules – 1 Measuring dish – 1 Sheet of Instructions for Use

The information on our products is based on comprehensive measurements taken during production, research and application engineering. We reserve the right to make minor technical changes in the development of the product.

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