- (DE) Gebrauchsanweisung
- (GB) Instructions for use
- (FR) Mode d'emploi
- (ES) Instrucciones de uso
- (IT) Istruzioni per l'uso
- (PT) Instruções de uso
- (NL) Gebruiksaanwijzing
- (SE) Bruksanvisning
- (DK) Brugervejledning
- (FI) Käyttöohjeet
- (GR) Οδηγίες χρήσης

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Produktbeschreibung

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GB Instructions for use Paladur®

Product description

Paladur is an autopolymerising prosthetic resin for all the indications listed below. It consists of a powder component in a range of colours (pink, pink veined, R50 veined, clear) and liquid. Prosthetic resin for casting or remodelling removable prosthetic devices that cures in thin layers on contact with air without technical efforts.

Composition

Cold-curing polymer, consisting of powder and liquid. Main powder component: Methyl methacrylate copolymer. Main liquid components: methyl methacrylate, 2-(2H-Benzotriazol-2-yl)-4-methylphenol. The catalyst system contains tertiary amine. Paladur is cadmium-free.

Indications

For fixed and removable prosthetic devices.
Can be used for:
- Restoring function
- Extension
- Additions
- Repairs
- Indirect partial and total underlays

Contraindications

Direct underlays with unpolymerised prosthetic material are contraindicated. If any known allergies exist to resins based on methacrylates or acrylates, the product should not be used.

Note!

Only for processing in dental laboratories in accordance with intended use.
When handling Paladur, do not use alcohol-based separating fluids (to separate plaster and wax) because they may cause white stains on the prosthesis.

Handling

Dental stone is recommended for the production of models. When performing extensions or repairs, use hot water to remove residual wax carefully from the resin surfaces to be bound. Bound rough areas on the resin parts to be bound (e.g., when repairing a bridge); no mechanical retention is required. The wet plaster model is insulated twice using thin layers of Aislar® (see Aislar® Instructions for Use). This is the only way to ensure perfect surface quality between Paladur and the model. To improve the bond between Paladur and prosthetic resins or the prefabricated teeth, the contact surfaces are roughened with a coarse diamond bur (removal grinding dust) and wetted with Palabond®. Apply Palabond® twice using a brush (without metal ferrule) and allow to set for 30 seconds each time. After the second application the adhesive remains active for 10 minutes.

Amounts required

The recommended mixing ratio is 10 g powder : 6 ml liquid. Introduce the liquid to the mixing beaker and add the appropriate quantity of powder rapidly and stir for up to 15 seconds to make homogeneous paste. To remove air bubbles, swirl the mixing beaker. Working time depends on ambient temperature. The drippy liquid paste can be worked immediately. Apply Paladur evenly and once the plastic consistency has been attained, do not mix or model further.

Safety information liquid

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Do not breathe dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. If skin irritation or rash occurs: Get medical advice/attention. In case of fire: Use for extinction: CO₂, sand, extinguishing powder. Store in a well-ventilated place. Keep container tightly closed.

Safety data sheets and more information are available at our website www.kulzer.com

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© = eingetragenes Warenzeichen der Kulzer GmbH
® = registered trademark of Kulzer GmbH