Palapress® vario

- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso
- PT** Instruções de uso
- NL** Gebruiksaanwijzing
- SE** Bruksanvisning
- DK** Brugervejledning
- GR** Οδηγίες χρήσης
- RU** Инструкция по применению

KULZER
MITSUI CHEMICALS GROUP

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

Distributed in USA /
Canada exclusively by:
Kulzer, LLC
4315 South Lafayette Blvd.
South Bend, IN 46614-2517
1-800-431-1785

Caution: Federal law restricts this device to sale by or on the order of a dental professional.

Instructions for use Palapress® vario

Product description

Palapress vario is an autopolymerising prosthetic resin for all the indications listed below. It consists of a powder component in a range of colours (pink, pink opaque, pink veined, R50 veined, clear, light pink, shade 200) and liquid. Palapress vario is suitable for use in the casting process for fixed and removable prosthetic devices.

Composition

Main powder component: Methyl methacrylate copolymer, dibenzoyl peroxide. Main liquid components: Methyl methacrylate, dimethacrylate. Catalyst system is free of tertiary amine. Palapress vario is cadmium-free.

Indications

Casting process:

- Completion of cast model prostheses
- Marginal shaping
- Repairs
- Indirect relines

Contraindications

Direct relines with unpolymerised prosthetic material are contraindicated. If the patient is allergic to one or more ingredients of Palapress vario, the product must not be used.

Advice

The product contains (meth) acrylate compounds that may cause sensitisation by skin contact. If any irritation appears or known allergies exist with resins based on methacrylates or acrylates, the product should not be used.

When setting up teeth in wax, do not use alcohol-based separating fluids (to separate plaster and wax) because they may cause white stains on the basal surface of the prosthesis. Only for processing in dental laboratories in accordance with intended use.

Casting procedure

Preparation

Dental stone is recommended for the production of models. The prefabricated teeth set up in wax are fixed with plaster or silicone (outer rim). After removal of the wax with hot water (without chemical additives), the stone model is insulated twice with thin layers of Aislar® (see Aislar® Instructions for Use). This is the only way to ensure perfect surface quality between Palapress vario and the model. To improve the bond between Palapress vario and the prefabricated teeth, the basal surfaces are roughened with a coarse diamond bur (remove grinding dust) and wetted with Palabond®. Apply Palabond® twice using a brush (without metal ferrule), and allow to set for 30 seconds each time. After the second application, the adhesive remains active for 10 minutes.

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Dosing and handling

The recommended mixing ratio is 10 g powder : 7 ml liquid. Introduce the liquid to the mixing beaker, add the appropriate quantity of powder rapidly and stir for up to 15 seconds to make a homogeneous paste. Prevent bubbles from forming. To remove air bubbles, swirl the mixing beaker. Working time depends on ambient temperature. After mixing, Palapress vario can be used for casting for approx. 3 minutes at a room temperature of 23°C (73°F), and after 4 minutes a plastic phase is reached lasting approx. 7 minutes.

Polymerisation

Polymerisation in the Palamat® elite/Palamat® practic should be performed after 7 minutes at the earliest and at the latest 13 minutes. The surface of the paste should have a dull appearance. Polymerisation time in the pressure vessel is 15 minutes, water temperature 55°C (131°F), pressure 2 bar.

Trimming and polishing

After checking and correcting occlusion, the prosthesis is removed from the model and prepared with cross-toothed cutters. Before polishing with pumice, the prosthesis is smoothed with sandpaper of decreasing grit size. **To further reduce the risk of allergic reactions, soak the prosthesis in lukewarm water for at least 12 hours before delivery (new fabrication or repair).**

Repair

Items made of Palapress vario can be restored and expanded with all Pala cold-curing polymers. The heat-curing Pala prosthetic resins from Kulzer can also be restored and expanded with all cold-curing Pala prosthetic resins.

Storage advice

Do not store above 25°C (77°F). Do not use after expiry date. Avoid direct exposure to sun. Reseal carefully after removing product. Keep away from children. Visually inspect for damage before first use. Damaged products must not be used.

Hazard/Safety information liquid

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. Avoid breathing dust/fume/gas/mist/vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Store locked up.

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Safety data sheets and more information are available at our website www.kulzer.com

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® = registered trademark of Kulzer GmbH
® = geregistreerd handelsmerk van Kulzer GmbH
® = registreret varemærke tilhørende Kulzer GmbH