
| | | |
|-----------|--------------------------------|----|
| DE | Gebrauchsanweisung..... | 5 |
| GB | Instructions for use | 9 |
| FR | Mode d'emploi | 13 |
| ES | Instrucciones de uso | 17 |
| IT | Istruzioni per l'uso | 21 |
| PT | Instruções de uso | 25 |
| NL | Gebruiksaanwijzing | 29 |
| SE | Bruksanvisning | 33 |
| DK | Brugervejledning | 37 |
| NO | Bruksanvisning | 41 |
| FI | Käyttöohjeet | 45 |
| GR | Οδηγίες χρήσης | 49 |
| CZ | Návod k použití | 53 |
| HU | Használati utasítás | 57 |
| LV | Lietošanas instrukcija | 61 |
| LT | Naudojimo instrukcija | 65 |
| PL | Instrukcja obsługi | 69 |
| RU | Инструкция по применению | 73 |

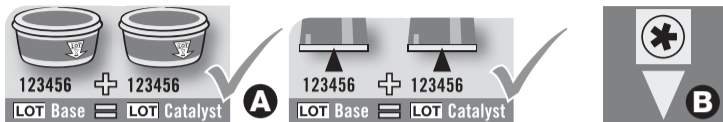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Elastomeric, silicone-based precision impression material, addition-cured

All products in the Provil *novo* assortment have a dosing ratio 1 : 1 (volume or weight).

Practical tips

Provil novo assortment is a variable consistency system that can be combined individually for all impression taking techniques. We do not recommend use in combination with any other silicone impression material. Visually inspect for damage before first use. Damaged products must not be used. Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. No allergic reactions have been demonstrated; however the product may still generate an allergic reaction. For use by dental personnel only.

Tubes and Tubs must be carefully sealed after use. Do not inter-change lids, caps or spoons. When using cartridges extrude some material before positioning mixing tip. After use leave used mixing tip in situ as sealer on cartridge. Before next use remove sealed tip. Extrude some material and then fix new mixing tip.

As is common with all addition cured silicone impression materials inhibiting reactions with catalyst paste can occur when in contact with latex gloves, retraction cords or solutions and newly placed composite fillings. Therefore, please ensure that patient's mouth (i.e. preparation and gingival margin) is thoroughly washed and dried before impression taking commences.

| | EN ISO 4823 | Product colour | | Mixing time |
|--|--------------------------------|----------------|------------|-------------|
| | | Base | Catalyst | s |
| Putty <i>fast set</i> | Type 0 putty | green | light grey | 30 |
| Putty <i>regular set</i> | | green | | 30 |
| Putty soft <i>fast set</i> | | turquoise | | 30 |
| Putty soft <i>regular set</i> | | turquoise | | 30 |
| Monophase <i>Tube</i> | Type 2 medium bodied | blue | light grey | 30 |
| Monophase CD 2 <i>fast set</i> | | blue | | – |
| Monophase CD 2 <i>regular set</i> | | light blue | | – |
| Medium <i>Tube</i> | Type 2 medium bodied | yellow | light grey | 30 |
| Medium CD 2 <i>fast set</i> | | yellow | | – |
| Medium CD 2 <i>regular set</i> | | light yellow | | – |
| Light <i>Tube</i> | Type 3 light bodied | dark green | light grey | 30 |
| Light CD 2 <i>fast set</i> | | dark green | | – |
| Light CD 2 <i>regular set</i> | | green | | – |

*at room temperature of 23°C (73°F), 50% rel. humidity.

Higher temperatures shorten, lower temperatures prolong these times.

| Extraoral working time* 23°C (73°F) min:s | Intraoral working time 35°C (95°F) min:s | Min. time in mouth min:s | Accessories | |
|---|--|--------------------------------|-----------------|-----------------|
| | | | Mixing Tips | Intraoral Tips |
| 1:30 | – | 2:30 | – | – |
| 2:00 | – | 3:30 | – | – |
| 1:30 | – | 2:30 | – | – |
| 2:00 | – | 3:30 | – | – |
| 2:30 | 1:00 | 3:30 | – | – |
| 1:30 | 0:40 | 2:30 | green 66001989 | white 65687514 |
| 2:00 | 1:00 | 3:30 | green 66001989 | white 65687514 |
| 2:30 | 1:00 | 3:30 | – | – |
| 1:30 | 0:40 | 2:30 | yellow 66001988 | yellow 66000782 |
| 2:00 | 1:00 | 3:30 | yellow 66001988 | yellow 66000782 |
| 2:30 | 1:00 | 3:30 | – | – |
| 1:30 | 0:40 | 2:30 | yellow 66001988 | yellow 66000782 |
| 2:00 | 1:00 | 3:30 | yellow 66001988 | yellow 66000782 |

Side effects:

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Model fabrication:

The impression can be cast 60 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

Recommended Model material:

All standard model materials can be used.

We recommend:

Class III Gypsum, e. g. MOLDANO®;
in the U.S.: Modern Materials®, Die Keen®;
Class IV Gypsum, e. g. MOLDASTONE®/MOLDASYNT®

Electroplating:

Impressions can be electroplated with copper or silver in conventional baths.

Disinfection:

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.
* (DGZMK 3/93 V1.0)

Adhesion:

We recommend using our Universal Adhesive.

Storage conditions:

Do not store above 25°C (77°F).

Dated: 2014-03

Description of applicational/ packaging icons

A = Only use pastes with same LOT No

B = Icon "Use of the special program at the Dynamix® speed Mixing Unit possible"

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Kulzer GmbH
Kulzer, LLC