(DE)	Gebrauchsanweisung	3
(GB)	Instructions for use	6
(FR)	Mode d'emploi	9
(ES)	Instrucciones de uso	12
(IT)	Istruzioni per l'uso	15
(PT)	Instruções de uso	18
(NL)	Gebruiksaanwijzing	21
(SE)	Bruksanvisning	24
(DK)	Brugervejledning	27
(NO)	Bruksanvisning	30
(FI)	Käyttöohjeet	33
(GR)	Οδηγίες χρήσης	36
(CZ)	Návod k použití	39

(HU)	Használati utasítás	42
(LV)	Lietošanas instrukcija	45
(LT)	Naudojimo instrukcija	48
(PL)	Instrukcja obsługi	51
(EE)	Kasutusjuhend	54
(HR)	Upute za uporabu	57
(RO)	Instrucţiuni de utilizare	60
(TR)	Kullanma talimatı	63
(KR)	사용 설명서	66
(BG)	Инструкции за употреба ..	69
(ID)	Petunjuk penggunaan	72
(SI)	Navodila za uporabo	75
(RS)	Uputstva za upotrebu	78

Signum metal bond

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Stand: 2016-01

Signum metal bond

Signum metal bond I+II is a metal resin bonding system for crown, bridge and CrCo framework.

Composition

Inorganic pigments, SiO₂ (silanised), methyl methacrylate, acetone, bifunctional methacrylate-based components

Indication

- Bonding methacrylate based resins / composites to metal framework surfaces in conjunction with macromechanical retention
- Bonding methacrylate based resins / composites to CrCo framework surfaces
- Bonding agent for repairing composite facings (intraoral and extraoral)
- Bonding agent for repairing porcelain bonded restorations
- Bonding between root posts, Maryland bridges and composite cement

Technique

Sandblast the surface to be faced with min. 110 µm Al₂O₃ at 3 bar pressure (non-recyclable abrasive). Then clean with uncontaminated (oil free) compressed air or clean tap water (do not steam clean) and dry thoroughly.

Dispense an ample amount of Signum metal bond I into a mixing dish. Apply to the prepared surface with a fine brush and allow to air dry. Apply a second coat if necessary.

Shake Signum metal bond II well to attain an optimal distribution of the colour pigments on the surface during application. Apply 1–2 uniform layers of Signum metal bond II with another brush and polymerize for 90 sec. with the Heraflash, HiLite power or UniXS. For intraoral use polymerize Signum metal bond II for 40 seconds with a dental polymerization lamp (e.g. Translux®).

For an optimum bond apply the relevant opaquer without undue delay and without contaminating the surface by touching it. Then apply the composite or acrylic.

Instructions and contraindications

Only for use by dentists and dental technicians for the above indications.

Close the Signum metal bond I and II bottles immediately after use.

Signum metal bond II should not cover the surface completely. The shade is masked by the application of Signum opaque F.

Hazard/Safety information

Signum metal bond I

Highly flammable liquid and vapour. Causes serious eye irritation. May cause drowsiness or dizziness. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. Avoid breathing dust/fume/gas/mist/vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing.

Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store locked up.

Signum metal bond II

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. Avoid breathing dust/fume/gas/mist/vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Store locked up.

Shelf life

The use by date is printed on the label of each bottle. Do not use after the expiry date.

Safety data sheets and more information are available at our website www.kulzer-dental.com

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Kulzer GmbH
Kulzer, LLC