Manufacturer:  
Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau (Germany)  
Made in Romania  
Distributed in USA/Canada exclusively by:  
Kulzer, LLC  
4315 South Lafayette Blvd.  
South Bend, IN 46614-2517  
1-800-431-1785  
Caution: Federal law restricts this device to sale by or on the order of a dentist.

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### Technical Information

#### Silicon based precision condensation curing impression material.

Optosil®/ Xantopren® Comfort®	EN ISO 4823	Colours	Dosage			Mixing-time	Working-time**	Time in mouth
			BASE	Activator universal Plus				
				Liquid	Paste			
Optosil® Comfort® Putty	Type 0	yellow	1 spoon (10,7g/9,5ml)	6-8 drops (0,14g/0,13ml)	6-8 graduations** (0,32g/0,29ml)	30	1:00	4:00
Xantopren® Comfort® Medium	Type 3	green	/	/	/	/	2:30	3:30
Xantopren® Comfort® Light	Type 3	blue	/	/	/	/	2:00	3:00

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Typ 0 = putty consistency  
Typ 3 = light bodied consistency

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### Lagerbedingungen:

### Warnhinweis zum Activator:

Flammbar. Inhalation kann cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. **Wear protective gloves/protective clothing/eye protection/face protection (EN 374, category III, e.g. Dermatri from KCL, Germany).** Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Do not use in pregnant or nursing patients or in children under 6 years of age. For use by dental personnel only.

### Pouring the model:

Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days. Visually inspect for damage before first use. Damaged products must not be used.

### Recommended model material:

All commercially available modeling stones can be used.  
Our suggestion:  
Class III Gips, MOLDANO®/MOLDADUR®  
Class IV Gips, MOLDASTONE®/MOLDASYN®

### Electroplating:

Impressions may be copper or silver plated in the usual way.

### Disinfection:

Impressions should generally be treated by immersion for 10 minutes\* in 5,25 % Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the products manufacturer.

### Adhesion:

We always recommend the use of our Universal Adhesive.  
Do not store above 25°C (77°F).

### Storage conditions:

### Warning Activator:

Flammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. **Wear protective gloves/protective clothing/eye protection/face protection (EN 374, category III, e.g. Dermatri from KCL, Germany).** Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Do not use in pregnant or nursing patients or in children under 6 years of age. For use by dental personnel only.

### Instructions for use

Optosil®/ Xantopren® Comfort®	EN ISO 4823	Colours	Dosage			Mixing-time	Working-time**	Time in mouth
			BASE	Activator universal Plus				
				Liquid	Paste			
Optosil® Comfort® Putty	Type 0	yellow	1 spoon (10,7g/9,5ml)	6-8 drops (0,14g/0,13ml)	6-8 graduations** (0,32g/0,29ml)	30	1:00	4:00
Xantopren® Comfort® Medium	Type 3	green	/	/	/	/	2:30	3:30
Xantopren® Comfort® Light	Type 3	blue	/	/	/	/	2:00	3:00

\*1 At room temperature of 23°C (73°F), 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.  
\*2 Graduation marks as indicated on the mixing pad and mixing spatula (1 graduation = 1.3 cm)  
The stated dosage should be observed.

### User Advice

The Optosil®/Xantopren® assortment is a complete system.

Prior to first use extrude some material from the cartridge. After use replace mixing tip by protecting cap (hygienics).

For best results usage with other silicon materials and Activators is not recommended.

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