






Indication 1 Cementation of Crowns

PANAVIATM V5

Zirconia

Clean and dry the tooth surface in the usual manner. As necessary, trial fit the crown using the Try-in Paste, wash and remove.

| | | | |
|---|---|--|---|
| 1 Blast with alumina powder, then ultrasonic clean and dry. | 2 Apply CERAMIC PRIMER PLUS and dry. | 3 Apply Tooth Primer, leave for 20 sec., and thoroughly dry with mild air. | 4 Dispense cement and place the crown. |
|  |  |  |  |

| | | | | |
|--|--|-----------|---|--|
| 5 Remove the excess cement using either method. Opaque shade is used for method B. | A. Tack-curing ① Light-cure for 3 to 5 sec.  ② Remove with a dental explorer.  | OR | B. Using a small brush ① Remove with a small brush.  ② Light-cure margins. Refer to table 2.  | 6 Maintain isolation for 3 minutes, or light-cure* for translucent crowns.  |
| | <p>* Refer to table 2. The opaque shade is self-cure only.</p> | | | |