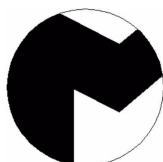
R-080075

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The R-080075 is a practical wax immerser heater which has been specially designed for use in the dental prosthesis sector. We recommend you read carefully the following instructions for use to ensure proper and lasting operation of this unit:

INSTALLING THE UNIT

- Carefully unpack the heater. Choose a flat surface for installation, well away from sources of heat and vibrations. Ensure also that the cooling branchiae to the rear of the unit have sufficient ventilation.
- Connect the unit to a 230 V, 50/60 Hz earthed power supply. You will observe “E80” on the display (preliminary microprocessor check) for approximately 3 seconds. Then “OFF” appears on the display, showing that the heater is in standby mode. The heater will now start up when the ON/OFF (C) connection key is pressed.

STARTING UP THE UNIT

The heater has a start-up routine for fast wax-melting when the wax is in solidified format. The routine consists of three stages, as follows:

- The first stage is heating the wax to 95 °C.
- Once this temperature has been reached, the control system retains it for approximately 1 minute in order to melt the wax completely.
- After the that minute have elapsed, the control system returns to normal operating mode, retaining the reference temperature.

During the start-up process, the control system display flashes to show instantaneous temperature, and thus the status. This start-up routine may be skipped by merely pressing the SET (D) key during execution. The display will then stop flashing, and the control system will move automatically to the reference temperature defined by the user. It should also be mentioned that the start-up routine activates only when temperature in the tank exceeds 50 °C. For lower temperatures, the control system knows that this is a start-up with melted wax, and so the routine is not executed.

OPERATION

1. When “OFF” shows on the control system, place the wax to be melted in the tank. Do not overfill the vessel to prevent overflows. The optimum quantity of wax could be $\frac{3}{4}$ of the vessel capacity, since this amount gets the maximum thermal efficiency.
2. If you want to set a reference temperature in this moment, press the “SET” (D) key and, simultaneously, press the up/down keys until the display shows the temperature required.
3. Press the “ON/OFF” (C) key. The heater will start up and show the instantaneous temperature of the vessel, and will also commence the start-up process (provided the vessel temperature is less than 50 °C). If you prefer the start-up routine not to be executed, press the “SET” key.
4. To select a reference temperature, simply press the “SET” (D) key and, simultaneously, press the up/down keys until the display shows the temperature required. Then release the “SET” key, and after 2 seconds the display will show the instantaneous temperature of the vessel again. As of this point the reference temperature will be retained for an indefinite period of time.
5. If, during operation of the unit, you wish to consult the reference temperature, simply press the “SET” (D) key. The display will then show the reference temperature for approximately two seconds.

6. During periods in which the unit is in disuse, the cover should be closed to prevent wax contamination and possible spillage.
7. When you wish to disconnect the heater, press the “ON/OFF” (C) key. The display will show “OFF”, the heating will be switched off and the control system will move to standby mode.

MAINTENANCE AND CLEANING

To ensure a long service life and optimum performance for this unit, we recommend the following steps be taken:

- On a regular basis, clean off the remains of wax from the vessel, the cover, and the outer casing. You may use a small spatula to carry out this operation. Be sure to disconnect the wax heater from the mains before carrying out such operations.
- Do not allow wax to drip onto the outer casing – proceed with caution.
- Do not use solvents to clean off wax stains. A sound method for cleaning wax off the interior of the vessel is as follows: when the wax is completely solidified, heat up the vessel. When the edges of the block of wax start to melt, disconnect the unit from the power supply, and extract the block using a small spatula. Any wax remains inside may be removed easily using a small chamois.

PRECAUTIONS TO BE TAKEN

- ◆ Ensure a suitable supply connection voltage (230 V, 50 /60 Hz, earthed).
- ◆ Do not use the unit in locations which are particularly damp, or in close proximity to inflammable materials (alcohol, solvents etc.).
- ◆ Do not put plastic objects into the vessel, or any other material which will melt at low temperatures.
- ◆ Always keep the cover closed when the unit is not being used.
- ◆ Wait at least 20 minutes after the unit has been disconnected before touching the vessel.
- ◆ Do not allow children or non-specialist staff to handle the unit.
- ◆ Do not allow wax to be spilt during operations, since its temperature could cause minor burns.
- ◆ Do not subject the mains cable to traction, or twist or bend it excessively.
- ◆ Before cleaning the unit, check that it has been disconnected from the mains.

TECHNICAL CHARACTERISTICS

Voltage:	230 V, 50/60 Hz
Power output:	30 W
Dimensions:	120 x 145 mm
Height:	90 mm
Maximum temperature:	97 °C
Vessel dimensions:	Ø 35 mm ↑ 35 mm
Weight:	0.6 kg

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