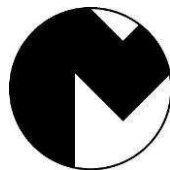
Ref. 080076

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The ref. 080076 High Capacity Wax Heater is a practical unit which keeps the wax temperature precisely as set by the user. The temperature range is between 30 and 100 °C. It has a number of applications in the dental sector. Please read the following instructions carefully for proper use of the machine, in order to obtain maximum output.

INSTALATION

1. Carefully unpack the unit. Check that it has not been damaged in transit.
2. Choose a flat and firm surface far from heat sources or vibrations.
3. Connect the machine to 230 V, 50/60 Hz with earthing.

HOW TO USE THE WAX HEATER

- Fill tank (C) with wax up to the minimum level (B), matching the horizontal line on the tank (approx. 30 mm from the upper rim).
- Activate the main switch at the back of the machine (on the supply cable base). Tank temperature will appear on the display. The machine **will start working immediatly**.
- If you wish to modify the temperature set (between 30 °C and 100 °C), press the [P] button for 3 seconds. Select the set temperature by pressing ▲/▼ keys. Press [P] to save.
- When the heating element is working, the pilot “OUT” lights.
- When set temperature is reached, the pilot “SET” lights.
- When the operation finishes, use the main switch on the back of the unit (on the supply cable base) to disconnect it.

MAINTENANCE AND CLEANING

- The machine has a protection device in case the level of wax in the tank falls below the

minimum level. If this happens, the heating element is disconnected automatically. Check the wax level regularly to prevent this.

- There is a fuse holder on the supply cable base. Change the fuse (4 A) whenever this is required.
- The machine should be cleaned periodically with a cloth. Before cleaning the machine, disconnect from the power supply.

PRECAUTIONS

- ◆ Before connecting the machine, make sure it is running at 230 V, 50/60 Hz and is earthed.
- ◆ When cleaning the machine, do not immerse it in any kind of liquid.
- ◆ Before cleaning the machine, disconnect from the power supply.
- ◆ Do not use the unit when the surroundings are too damp, or when it is too close to any inflammable products (alcohol or solvents).
- ◆ Do not pour solvents or any other inflammable product into the tank.
- ◆ Do not allow children or unqualified individuals to handle the machine.
- ◆ Do not start the machine if wax level is below the minimum. Check this level periodically, as wax may evaporate.

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GUARANTEE

All MESTRA® units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, MESTRA® will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA® will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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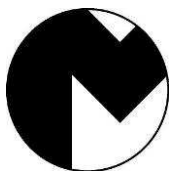
TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prosthesis sector, from the business address shown at the bottom of the page.

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I HEREBY STATE: That the **HIGH CAPACITY WAX HEATER ref. 080076**, with the following characteristics:

Alto / Height:	160 mm
Ancho / Width:	235 mm
Fondo / Depth:	200 mm
Peso / Weight:	1 kg
Capacidad / Capacity:	1 L
Alimentación / Power supply:	230 V, 50/60 Hz
Potencia / Power level:	310 W
Fusible / Fuse:	4 A
Rango de temperatura / Temperature range:	30-100 °C

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Meets the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.

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In witness whereof, the company issues this declaration of conformity in Sondika, on 03 September 2019.

Firma / Signed:

Ignacio Mestraitua,
Director Gerente / Managing Director

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA