R-080087

E.....	DECANTADORA CON BOLSA RECOGEDORA.....	6
GB....	DECANTER WITH COLLECTION BAG.....	7
F.....	BAC DE DÉCANTATION AVEC SAC RÉCUPÉRATEUR.....	8
D.....	GIPSABSCHIEDER MIT FOLIENEINSATZ.....	9
P.....	DECANTADORA COM SACO.....	10
I.....	DECANTATRICE CON SACCHETTO DI RACCOLTA.....	11

MESTRA®

TALLERES MESTRAITUA S.L.


Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 17/03/09

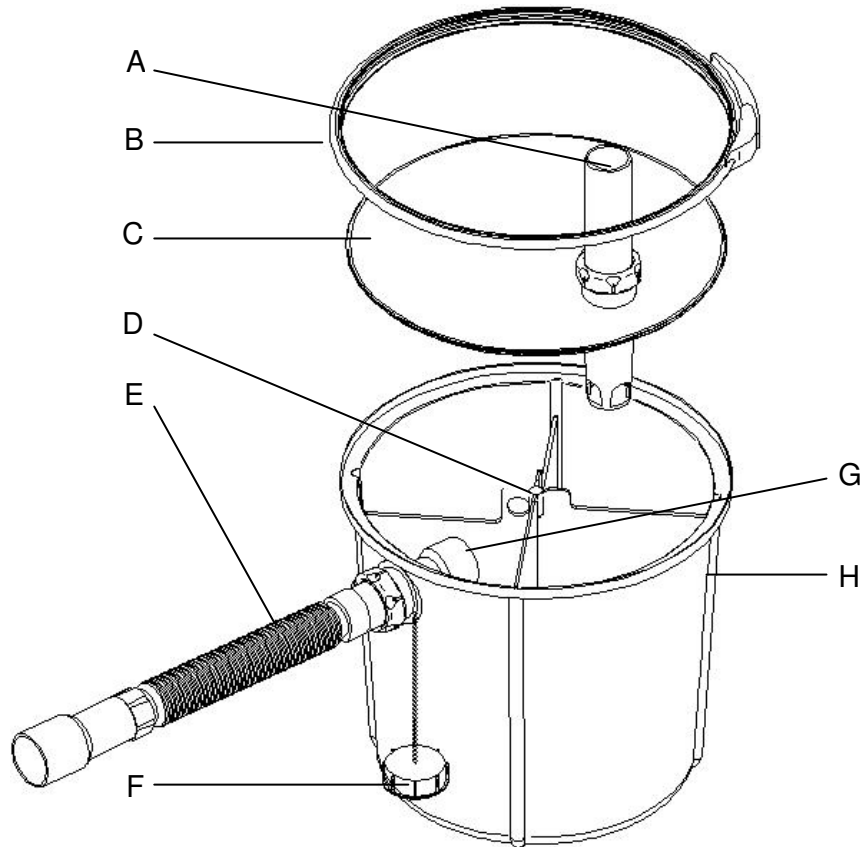R-080087

A	Entrada / Inlet / <i>Entrée</i> / Einlaufstutzen / Junção de entrada / <i>Ingresso</i>
B	Abrazadera / Clamp / <i>Anneau de cerclage</i> / Spannring / Braçadeira / <i>Ghiera</i>
C	Tapa / Lid / <i>Couvercle</i> / Deckel / Tampa / <i>Coperchio</i>
D	Separadores / Separators / <i>Séparateurs</i> / Trenneinsatz / Separadores / <i>Separatori</i>
E	Salida / Outlet / <i>Sortie</i> / Auslaufstutzen / Junção de saída / <i>Raccordo di uscita</i>
F	Tapón / Plug / <i>Bouchon</i> / Verschlusskappe / Tampa / <i>Tappo</i>
G	Adaptador del desagüe / Drainpipe adaptor / <i>Adaptateur d'écoulement</i> / Auslaufadapter / Adaptador do escoamento / <i>Adattatore scarico</i>
H	Cuba / Tank / <i>Cuve</i> / Behälter / Cuba / <i>Vaschetta</i>

R-080085-02

Bolsas de repuesto (20 unidades)
Spare bags (20 units)
Sacs de remplacement (20 unités)
 Ersatztüten (20 Stück)
Sacos de reserva (20 unidades)
Sacchetti di ricambio (20 unità)

1

2

3

4

5

6


CIERRE DE LA TAPA
CLOSING THE LID
FERMETURE DU COUVERCLE
SCHLIESSEN DES DECKELS
FECHO DA TAMPA
CHIUSURA DEL COPERCHIO

7

8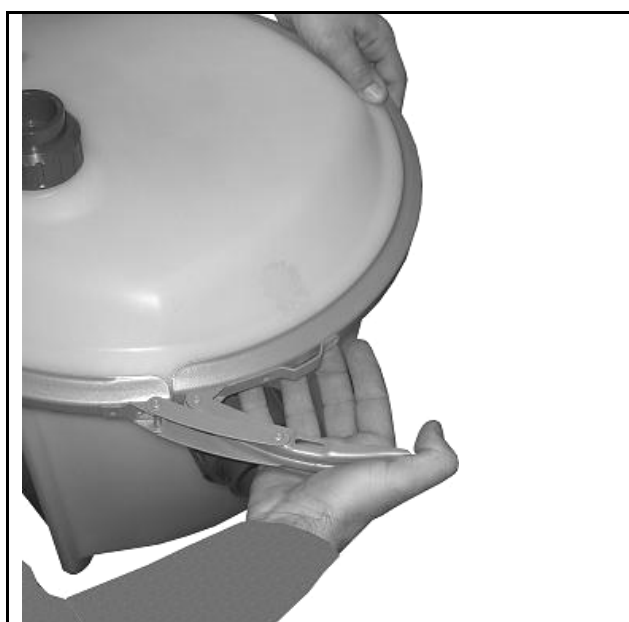

9


10

11

CAMBIO DE LA BOLSA
REPLACING THE BAG
REPLACEMENT DU SAC
WECHSEL DES FOLIENEINSATZES
SUBSTITUIÇÃO DO SACO
SOSTITUZIONE DEL SACCHETTO

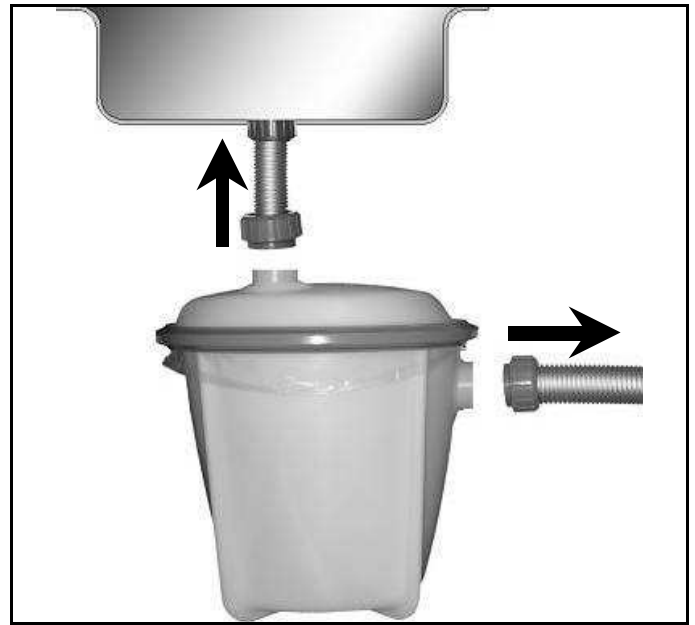
12

13

14

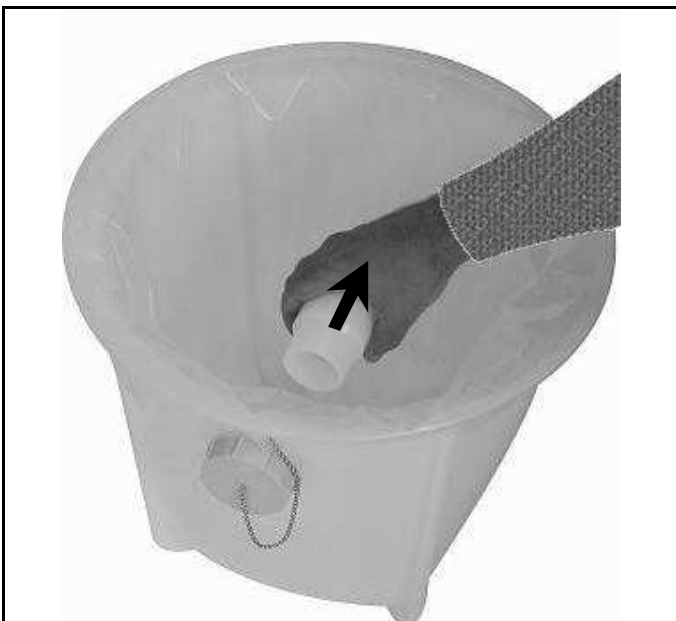15

16

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CARACTERÍSTICAS TÉCNICAS

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The R-080087 decanter is made of high resistance plastic and it is specially designed to allow a quick and simple cleaning, since dirt is collected in a plastic bag. The lid closes hermetically and prevents the spread of bad odours. Its size and shape have been thought for the unit to be easily installed under the sink.

INSTALLATION (see Figures in previous pages)

1. Insert the bag into the tank while folding the bag's rim outwards.
2. Insert the separators in the grooves down to the tank bottom. The decanter's outlet should be in the lowest chamber.
3. Place the bushing on the outlet hole and press it strongly while stretching the bag.
4. Open the drain hole in the bag with a knife.
5. Once the drain hole has been opened, press the bushing until it remains correctly fitted.
6. Place the lid making sure the water inlet is in the highest chamber.

In order to ensure an optimum operation of the decanter and to prevent deformations, the lid should be closed observing the following instructions:

7. Place the decanter's lid over the tank and make sure it is well centred and fitted all along its perimeter. Open the clamp completely and place it around the decanter.
8. Loosen the clamp's clasp and make sure the lid is well centred and fitted before tightening the clamp.
9. Tighten the clamp carefully to prevent the lid moving.
10. Close the clamp completely. The lid should be perfectly fitted to the tank.
11. Put the decanter under the sink. Connect the sink's outlet to the decanter's inlet and the latter's outlet to a drainpipe. In case of any trouble when adapting the decanter to the drainage system in your lab, consult a professional plumber.

CLEANING

12. To remove the decanter from under the sink, loosen the inlet and outlet threads.
13. Put the threaded plug on to close the drain hole. Open the lid and remove the remaining water.
14. Remove the separators.
15. Remove the adaptor from the outlet hole.
16. Tie up the bag below the drain hole and dispose of the waste.

CAUTION

- ◆ Place the decanter on a firm, flat and balanced surface.
- ◆ Make sure there are not electric wires or circuits near the decanter. A minimum safety distance of 1 meter is recommended.
- ◆ Be extremely cautious while transporting or moving the decanter. Prevent the risk of turning the unit over.
- ◆ In case of any trouble when adapting the decanter to the drainage system in your lab, consult a professional plumber.

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