
R-080102

E	RECORTADORA DE INTERIORES	3
GB	INSIDE TRIMMER	5
F	DÉTOUREUSE DE MODÈLES	7
D	INNENBESCHNEIDEGERÄT	9
P	MÁQUINA CORTADORA DE INTERIORES	11
I	TAGLIATRICE DI INTERNI	13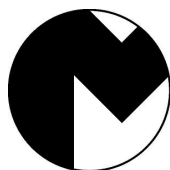

MESTRA®


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

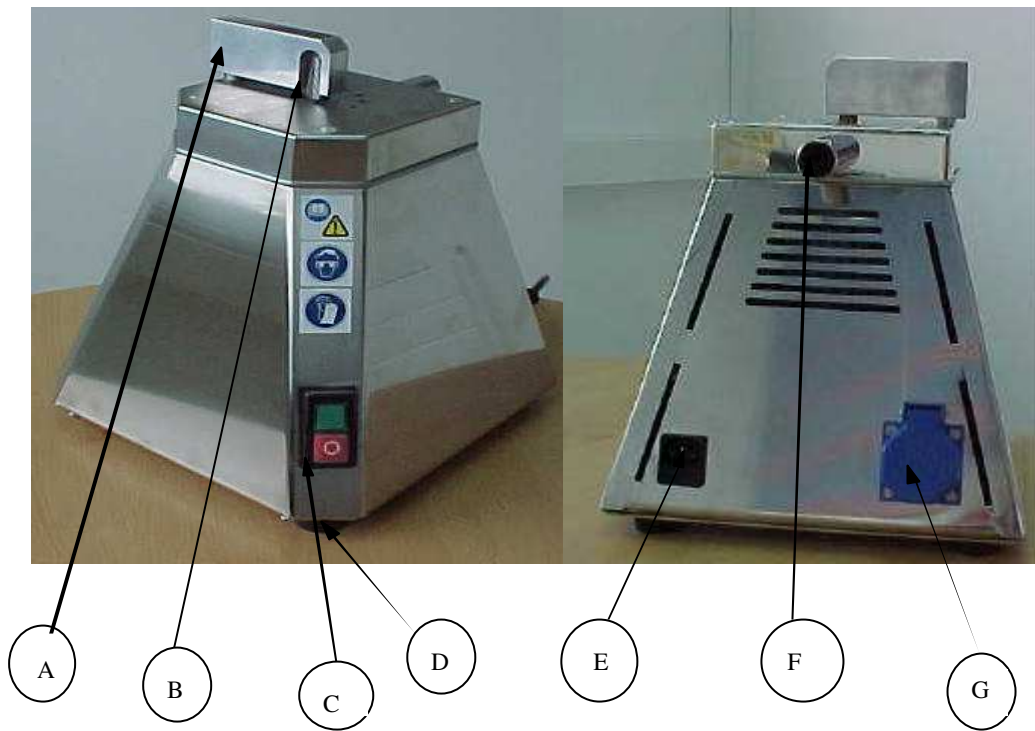R-080102

- | | |
|---|---|
| A | Pantalla de protección de fresa / Mill protection screen / <i>Protecteur de fraise</i> / Fräsenschutzschirm / Ecrã de protecção da fresa / <i>Schermo di protezione fresa</i> |
| B | Fresa / Milling unit / <i>Fraise</i> / Fräse / Fresa / <i>Fresa</i> |
| C | Interruptor general / Main switch / <i>Interrupteur général</i> / Hauptschalter / Interruptor geral / <i>Interruttore generale</i> |
| D | Tacos de goma / Rubber legs / <i>Pieds en caoutchouc</i> / Gummiunterlage / Tacos de goma / Tasselli di gomma |
| E | Fusible / Fuse / <i>Fusible</i> / Sicherung / Fusível / <i>Fusibile</i> |
| F | Boca de aspiración / Intake hole / <i>Tuyau pour aspiration</i> / Absaugöffnung / Boca de aspiração / <i>Bocca d'aspirazione</i> |
| G | Toma de corriente (Aspirador) / Current pickup (suction unit) / <i>Prise pour aspiration</i> / Stromanschluss (Absauger) / Tomada de corrente (aspirador) / <i>Presà di corrente (Aspiratore)</i> |

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The R-080102 is a simple inside trimmer for plaster which has been specially designed to ease trimming in the dental prosthesis sector. The trimmer is supplied ex-works with a 6-tooth tungsten mill cutter (B), and a guard screen (A).

The trimmer has a suction tube (F) (the unit is supplied with an adapter, Ø 35 mm) located to the rear of the machine. The tube leads directly to a chamber where powder is taken during cutting, via two holes on the operation table. A rubber gasket between the upper cover and the main trimmer body ensures a perfect seal between them, and so cutting waste may only move through the suction tube (F).

The appliance has a current pickup (G) at the rear to connect up to the suction system.

The machine has four fastening suckers (C). Particular care was exercised at the design stage to provide sufficient weight and sturdiness to prevent any movement during cutting operations, and so no further attachment devices are required.

CONDITIONS OF USE

We recommend the following instructions be observed for proper handling of the machine:

- Unpack the trimmer carefully. Check that it has suffered no damage in transit.
- The unit is supplied with the following accessories:
 - 3 mm Allen key.
 - 5 mm Allen key.
 - Suction tube adapter, Ø 35 mm.
- Connect the machine to an earthed supply of 220 V, 50/60 Hz.
- Before cutting a part, it is important a suction system be installed.
- Connect the suction system.
- Activate the main switch (C).
- You must have sufficient protection during operation of the trimmer, as follows:
 - Use safety goggles.
 - Use safety gauntlets.
 - Use a face-mask.
 - Always use the trimmer with the suction facility operational.
- Once operations have been completed, disconnect the main switch (C).
- After the trimmer has been used, we recommend cleaning the work surface with a damp cloth to remove any remains of plaster.

The machine has a protection system to prevent sudden start-ups. If this system were not installed, the trimmer motor would stop in the event of a power outage. However, when the power returned, the machine could then start up again more or less

unexpectedly, and an accident could occur if someone were handling the shafts of the motor at that time.

To prevent such risks, TALLERES MESTRAITUA, S.L. has included a switch for the trimmer which automatically disconnects the unit in the event of a power outage. If this should occur during a work session, the device will activate, and the unit must then be manually reset by using the switch again.

DISASSEMBLY OF THE MILLING MACHINE

The machine is equipped with a 6-tooth tungsten mill cutter. Proceed as follows to carry out the replacement operation:

1. Lift the work surface by loosening the four attachment bolts with a 3 mm Allen key. This position provides direct access to the milling unit.
2. Use a 4 mm Allen key to loosen the bolt holding the milling unit.
3. Release the mill unit, and replace if required. The mill shaft has a flat surface which must fit the screw.
4. To reassemble the milling unit, carry out these instructions in reverse order. Be careful to ensure that the rubber seal is correctly positioned.

MAINTENANCE AND CLEANING

To ensure proper, reliable and durable operation, we recommend the following:

- After the machine has been used, clean off the remains of plaster and particles on the surface or on the main trimmer body.
- Use a damp soapy cloth for general cleaning. Never use alcohol, solvents or other inflammable liquids.
- Never submerge the trimmer in liquids.
- Should abnormal noises or vibrations be detected, send the unit to the technical assistance unit.
- Ensure that the rubber fitting provides a perfect seal and does not let through any particles.
- When changing the mill unit, check that the trimmer works properly, with no abnormal vibrations.
- The machine has been designed for proper operation resting on the four rubber supports. These components can withstand the cutting force unaided.
- For best performance, Talleres Mestraitua, S.L. recommends the use of a suction system, since the inside trimmer unit has been specifically designed for this combination.
- If the machine does not start up, first check the condition of the fuse.

PRECAUTIONARY MEASURES

- ◆ Before connecting the machine to the mains, ensure that supply voltage is correct (230 V, 50/60 Hz).
- ◆ Check the condition and securing of the milling unit before commencing cutting operations.
- ◆ During operation of the unit, use sufficient protection - gauntlets, safety goggles and a face-mask.
- ◆ Do not allow children or non-specialist staff to handle the machine.
- ◆ If the motor shaft should lock out during normal operation of the unit, disconnect the machine from the mains before attempting to free the shaft by manual rotation.
- ◆ Do not trim materials which may leave particles which are toxic when inhaled.
- ◆ Before using the machine, ensure that it is resting on a firm and solid base.
- ◆ Do not use the machine in close proximity to solvents, alcohol or other inflammable materials. Extreme care should be taken when cutting parts which may cause sparks.
- ◆ Ensure that the rubber fitting provides a perfect seal and does not let through any particles.

TECHNICAL CHARACTERISTICS

Single-phase motor / Consumption:	250 W
R.P.M.:	2850
Voltage:	230 V, 50/60 Hz
Height:	300 mm
Width:	275 mm
Depth:	275 mm
Weight:	11 kg

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