## R-080501

E .....	PULIDORA PARA ASPIRACIÓN CENTRALIZADA.....	3
GB .....	POLISHER FOR CENTRALIZED ASPIRATION.....	6
F.....	BOX À POLIR POUR ASPIRATION CENTRALISÉE.....	9

**MESTRA®**


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

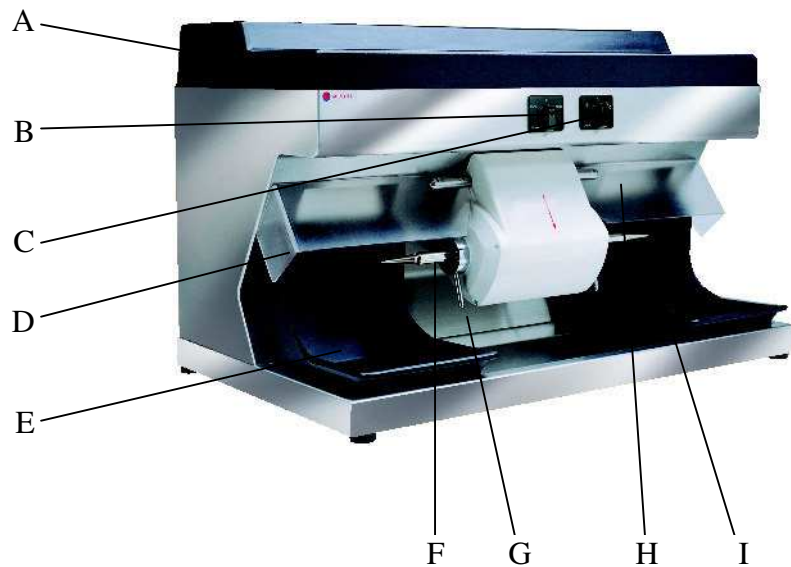## R-080501

A	Tapa / <b>Cover</b> / <i>Couvercle</i>
B	Selector de velocidad / <b>Speed selector</b> / <i>Selecteur de vitesse</i>
C	Interruptor de marcha / <b>On switch</b> / <i>Interrupteur marche-arret</i>
D	Visera protectora / <b>Protective visor</b> / <i>Visiere de protection</i>
E	Pantalla / <b>Screen</b> / <i>Augette</i>
F	Portacepillos a Izda / <b>Left-hand brush-holder</b> / <i>Porte-brosses gauche</i>
G	Mango de extractor / <b>Extractor bushing</b> / <i>Manche d'extracteur</i>
H	Entrada de aire / <b>Air inlet</b> / <i>Entree d'air</i>
I	Portacepillos a Dcha. / <b>Right-hand brush-holder</b> / <i>Porte-brosses droit</i>

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The R-080501 is a complete polisher equipped with safety devices to European EC standards. It is especially designed to polish and grind small workpieces for the dental prosthesis sector. It is supplied from the factory with the following accessories:

- 1 left-hand thread brush-holder (F)
- 1 right-hand thread brush-holder (H)
- 1 right-hand thread stone holder (not shown in figure)
- 2 rubber screens

As an option the unit can be fitted with a mill holder (Mestra catalogue ref. R-080190) for 2.35 mm shank milling cutters. Left-hand brush-holders and right-hand wheel holders are also available.

### INSTALLATION

- Carefully unpack the polisher.
- Set up the unit on a flat surface away from any sources of heat or vibration. A well ventilated area should be chose.
- Connect the unit to a 230 V, 50/60 Hz earthed power outlet.
- Mount the brush and/or grinding wheel on the motor shafts.

### MOUNTING THE BRUSH IN THE BRUSH-HOLDER

To attach the brush correctly to the brush-holder (F) proceed as follows:

1. Choose a suitable brush-holder. For mounting on the right-hand end of the motor shaft as seen from the front of the unit, choose a right-hand (clockwise) thread brush-holder so that the friction torque tightens it during operation. Likewise, for mounting on the left-hand end of the unit choose a left-hand (anti-clockwise) thread brush-holder.
2. Attach the unthread part of the brush-holder to a bench vice.
3. Insert the brush into the brush holder, ensuring that the axis of the brush-holder is perpendicular to the plane of the brush.
4. Thread the brush onto the brush-holder as tightly as possible by hand.
5. Take the holder and brush assembly out of the bench vice.
6. Visually check the perpendicularity of the brush-holder axis to the plane of the brush by spinning the assembly in your hands.

### MOUNTING GRINDING WHEELS ON THE WHEEL-HOLDER

1. Select the wheel-holder and remove the wheel attachment screw from the end.
2. Secure the other end of the wheel-holder to a bench vice.
3. Mount the wheel in the wheel-holder.
4. Secure the wheel in the wheel-holder by tightening the wheel attachment screw with a cross-head screwdriver.
5. Remove the wheel-holder/wheel assembly from the bench vice.
6. Visually check the perpendicularity of the wheel-holder axis to the plane of the wheel by spinning the assembly in your hands.

### MOUNTING THE BRUSH-HOLDER AND WHEEL-HOLDER ON THE SHAFT OF THE UNIT

1. Place the end of the wheel-holder or brush-holder with the hole on the end of the motor shaft and push it on firmly by hand.
2. Tap the other end of the wheel-holder or brush-holder with a nylon mallet until it is properly fitted on.
3. Turn the motor shaft manually and check visually that the grinding wheel or brush is perpendicular to the axis of rotation. Also check the alignment between the motor shaft and the axis of the wheel-holder or brush-holder.
4. Start up the unit so that the motor spin shows up any tilt in the wheel or brush. If tilt or more than 1 or 2 mm is observed, remove and re-mount the wheel-holder or brush-holder.
5. Immediately after mounting a brush in the brush-holder, remember that the friction torque helps tighten it. Set the attack angle on the workpiece to be brushed so as to favour this effect.
6. The safety visors (D) are made of polycarbonate and are virtually unbreakable. They can be folded down for easier use. Raise them when mounting the brush-holder and wheel-holder for easier access to the motor shaft.

## DISMOUNTING THE BRUSH-HOLDER AND WHEEL-HOLDER FROM THE SHAFT OF THE UNIT

The unit comes with two extractors (G) to help remove the brush-holder or wheel-holder. Grip the handle of the extractor firmly and turn it anti-clockwise (right hand end of shaft) or clockwise (left-hand end). The pressure exerted by the extractor on the brush-holder will smoothly release them.

## USING THE UNIT

To ensure correct use of the polisher, follow these instructions:

- Mount a suitable brush-holder or wheel-holder as described above.
- Lower the safety visor (D) and ensure that the rubber screens (E) are in place. If they are incorrectly positioned hazards can result and the machine's efficiency is impaired.
- Before starting to polish or grind a workpiece, ensure that the motor spin direction is correct (red arrow on motor stator), as this affects the initial angle of attack on the workpiece. Incorrect angles of attack can be highly dangerous.
- Start up the polisher via switch (B). Turn the switch right or left to select the desired spin speed (1500-3000 at 50 Hz or 1800-3600 at 60 Hz).
- Turn switch (C) right to activate the light. The following protective equipment is recommended for polishing or grinding:
  - safety goggles;
  - safety gauntlets;
  - face-mask;
  - As a general rule do not use the polisher without the exhauster.
  - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
  - Be careful with long hair.
  - No objects similar to ropes or chains that could be hooked by the machine.
  - Place the protective screen properly.
  - Work with the visor (D) in place.
- After using the unit always clean the inside of the protective screen and the support tray.

## MAINTENANCE AND CLEANING

To ensure correct, reliable, long-lasting operation of the unit we recommend the following:

- Clean the dust off the inside of the protective screen with a brush after each use. Also clean the surface of the front tray.
- Regularly clean the screens thoroughly. Also clean the stainless steel outer casing of the polisher with a rag moistened in soapy water.

- Use a rag moistened in soapy water also to clean the motor and support. Never use alcohol, solvents or other inflammable liquids.
- The unit has a protective system to prevent sudden start-ups. If the electrical power supply to the polisher motor is cut off it stops, but when power is restored the machine might start up unexpectedly and anyone manipulating the motor shafts at the time might suffer injury. To prevent this Talleres Mestraitua, S.L. has fitted the R-080501 with an automatic switch which turns it off in case of a power failure. If this happens during operation the device will have to be reset before work can continue. Do this as follows:
  1. Switch off the unit via switch (B).
  2. Raise the top dust guard (A).
  3. Locate the automatic off switch, between switches (B) and (C).
  4. Reset the automatic off switch by pressing it.
  5. Fit the dust guard (A).

## PRECAUTIONS

- Before connecting the unit ensure that the power supply is 230 V, 50/60 Hz earthed power outlet.
- Discard any brush or grinding wheel with cracks, knocks or flaws. Breakage of these components during operations could cause serious accidents.
- Before starting work ensure that the brush-holder/wheel holder is firmly coupled to the motor shaft.
- When handling workpieces use safety goggles, gauntles and mask.
- Do not allow children or unskilled personnel to handle the unit.
- Never polish, brush or grind materials which might give off particles which are toxic if inhaled.
- Before starting to polish or grind a workpiece, ensure that the motor shaft spin direction is correct (red arrow on motor stator). Attack angle depends on spin direction.
- Do not use the unit near solvents, alcohol or other inflammable materials. Take extreme care when grinding materials which may give off sparks.
- Before using the unit ensure that it is set up on a firm solid base.
- Do not use the unit without the exhauster unless the nature and characteristics of a specific job require that it be disconnected. In this case keep working time as short as possible.

**TECHNICAL CHARACTERISTICS**

Height:	500 mm
Width:	800 mm
Depth:	500 mm
Weight:	60 kg
Power supply:	230 V, 50/60 Hz
Speed (rpm):	1500/3000, 50 Hz 1800/3600, 60 Hz
Noise level:	60 dB
Exhauster flow rate:	500 m <sup>3</sup> /hora

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