
R-080600 ALTAMIRA

E.....	BOX PARA VAPORETO.....	7
GB	STEAM CLEANING BOX	10
F.....	BOX POUR MACHINE A VAPEUR.....	13
D	BOX FÜR VAPORETO	16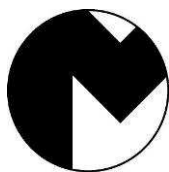

MESTRA®


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60


Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA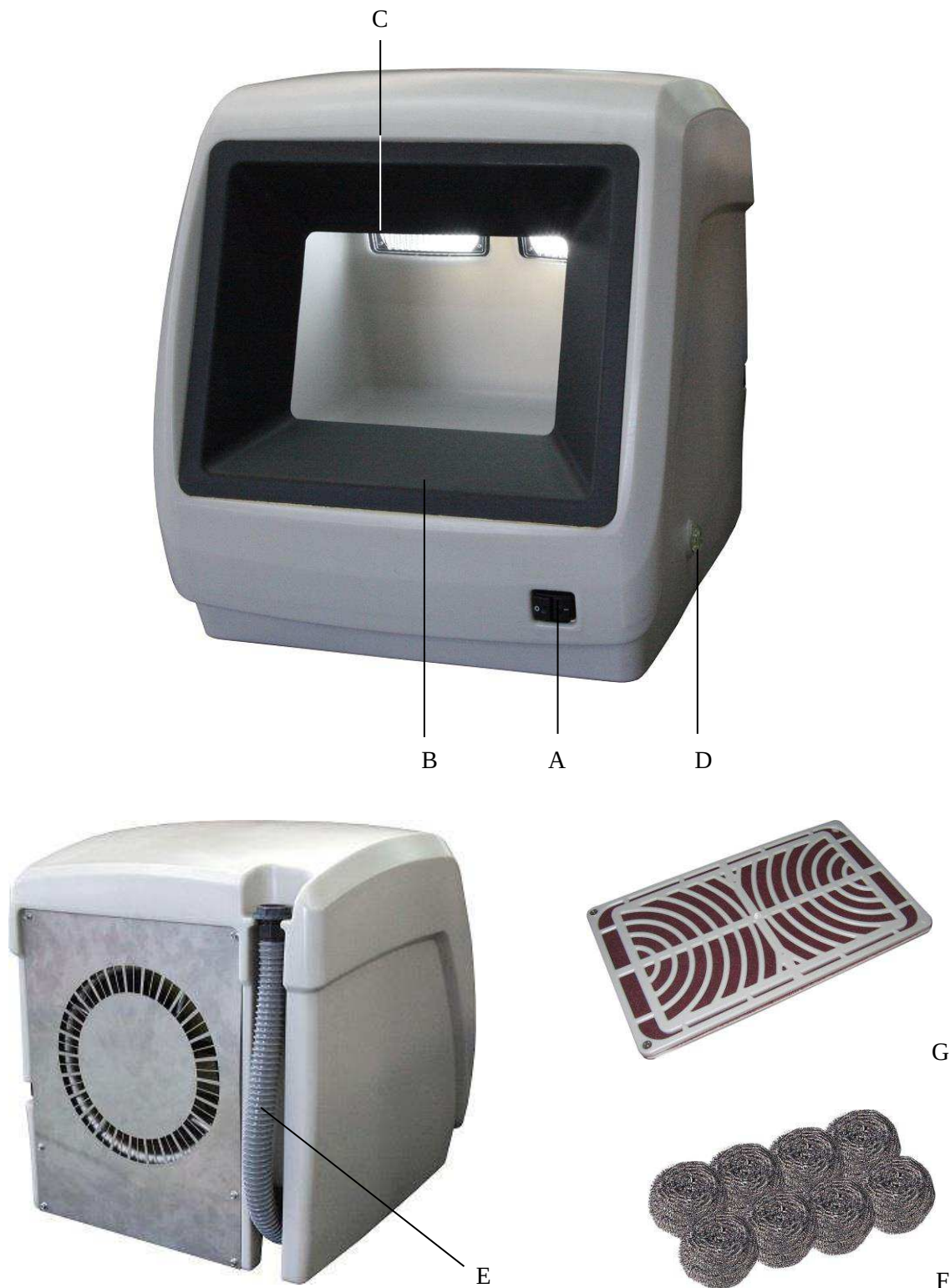

- | | |
|----|---|
| A. | Interruptor general / Main switch / <i>Interrupteur général</i> / Hauptschalter |
| B. | Marco frontal / Difusor frame / <i>Façade</i> / Rahmen Vorderseite |
| C. | Luz / Light / <i>Éclairage</i> / Licht |
| D. | Visor nivel de agua / Water level visor / <i>Viseur niveau d'eau</i> / Füllstandanzeige |
| E. | Manguera de desagüe / Outlet hose / <i>Tuyau de vidange</i> / Schlauch für Wasserabfluss |
| F. | Filtro metálico / Metal filter / <i>Filtre métallique</i> / Metallfilter |
| G. | Filtro de tela / Cloth filter / <i>Filtre en toile</i> / Stofffilter |

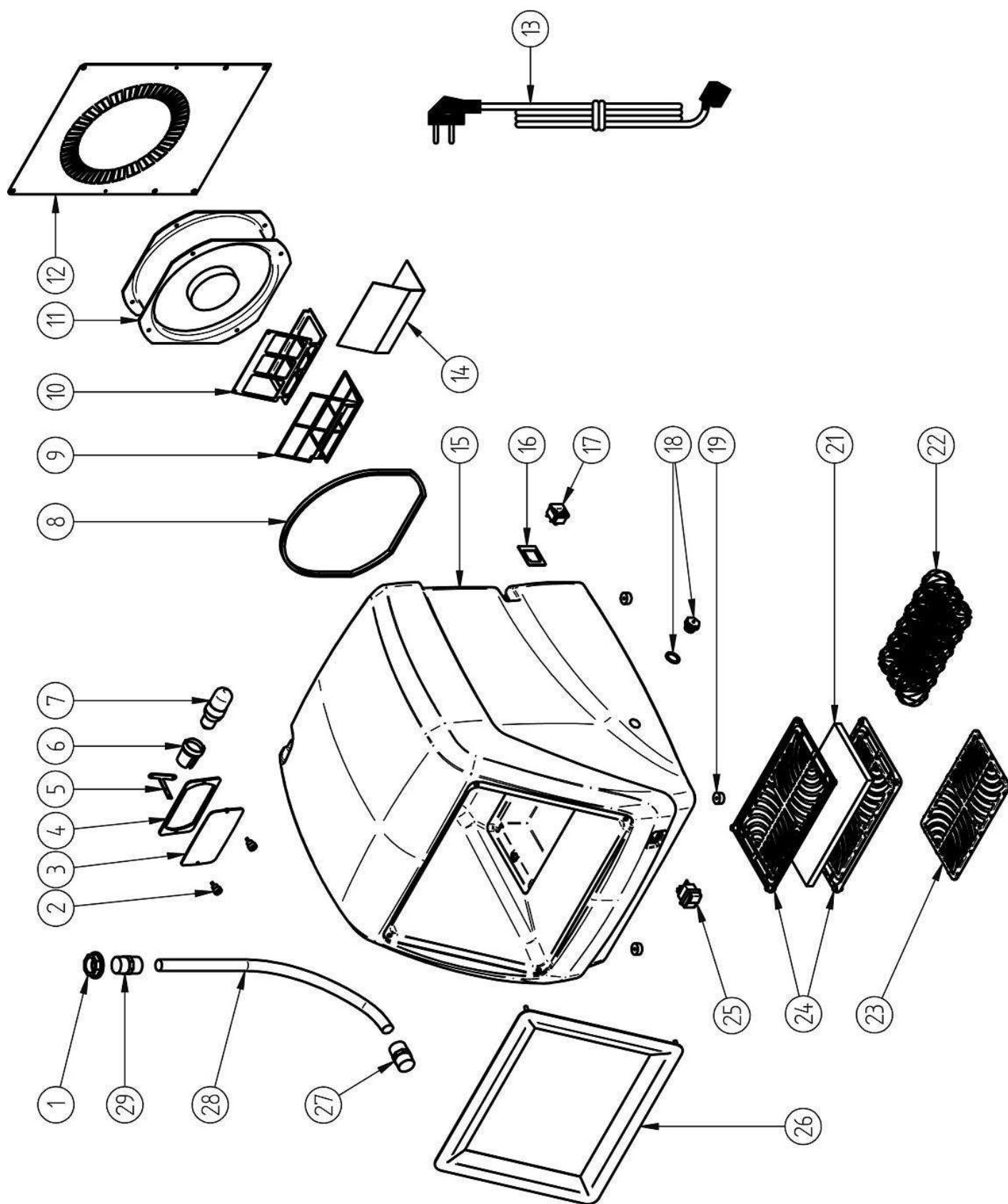Nº	Ref.	CONCEPTO	DESCRIPTION
1	080086-06	Tuerca de manguito	Sleeve nut
2	080600-28	Pomo pantalla	Shield knob
3	080600-02	Pantalla lámpara	Lamp shield
4	080600-07	Junta pantalla	Shields gasket
5	080600-16	Soporte portalámparas	Lamp holder support
6	080600-13	Portalámparas	Lamp holder
7	080600-05	Lámpara 230V	Lamp 230V
	080600-05B	Lámpara 115V	Lamp 115V
8	080600-26	Junta de ventilador	Fan gasket
9	080600-19	Marco filtro ventilador	Fan filter frame
10	080600-20	Empujador filtro ventilador	Fan filter pusher
11	080600-03	Ventilador 230V	Fan 230V
	080600-03B	Ventilador 115V	Fan 115V
12	080600-18	Chapa trasera	Back sheet
13	000250	Cable 230V	Cable 230V
	000252	Cable 115V	Cable 115V
14	080600-21	Filtro de espuma	Foam filter
15	080600-08	Carcasa	Housing
16	080600-17	Soporte base porta fusibles	Fuse holder base support
17	080490-06	Base porta fusibles	Fuse holder base
18	080600-01	Visor	Visor
19	080490-31	Patas	Legs
21	080600-22	Filtro estropajo	Cloth filter
22	080600-12	Conjunto filtros metálicos	Metal filter kit
23	080600-40	Rejilla inferior cámara	Chamber lower grid
24	080600-39	Rejillas superiores de cámara	Chamber top grids
25	080400-07	Interruptor	Switch
26	080600-09	Marco frontal	Front frame
27	080086-07	Manguito inferior de manguera	Lower hose sleeve
28	080600-04	Manguera de desagüe	Hose
29	080210-02	Manguito superior manguera	Top hose sleeve

MESTRA®

Your trusted brand

TALLERES MESTRAITUA, S.L.

Txori-erri Etorbidea, 60

Tel. + 34 944 530 388* - Fax + 34 944 711 725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - (España)

MESTRA®

Your trusted brand

TALLERES MESTRAITUA, S.L.

Txori-erri Etorbidea, 60

Tel. + 34 944 530 388* - Fax + 34 944 711 725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - (España)

MESTRA®

Your trusted brand

TALLERES MESTRAITUA, S.L.

Txori-erri Etorbidea, 60

Tel. + 34 944 530 388* - Fax + 34 944 711 725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - (España)

**MESTRA®****GARANTIA**

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**GUARANTEE**

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA® will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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The R-080600 box has been specially designed to work together with a steam cleaner, and to collect dirt and condensed water. It has a wide working camera perfectly lighted, and an integrated extracting system to remove steam. It is made of high quality plastic, and it is easy and simple to clean. The noise level of the extracting system is low and it is not annoying.

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.


MEANING OF THE SYMBOLS ON THE UNIT

Danger. Risk of injury.

Read completely these instructions before operating the unit.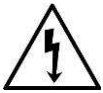

Electric shock risk.

INSTALLATION

Environmental conditions:

- Use indoors only.
- Environmental temperature: 5-40 °C [41-104 °F]
- Max. relative humidity: 0-95 %
- Degrees of protection: IP20

1. Carefully unpack the box and check it has not suffered any damage during its transport.
IMPORTANT: Do not hold the box by the front frame (B). Due to the weight of the machine, the frame may unstick (Fig. 1).
2. Place the device on an horizontal surface, flat and firm, away from heat or vibration sources. Ensure especially that the back ventilation grill is free and at least 15 cm from the closest object or wall.
3. Place the device so that it is easy to use the disconnecting device (main switch).
4. Connect the device to a grounded 230 V, 50/60 Hz power source.

Fig. 1

OPERATION

1. Switch the main switch on, which is placed in the front of the machine.
2. Two lights in the chamber and the fan will start.
3. Clean the pieces with the steam machine. The fan will aspirate steam avoiding it go out and improving the work vision.

MAINTENANCE AND CLEANING

To ensure long, reliable operation of the unit, proceed as follows:

- The box has a capacity for 8 liters of condensed water. Check level no reach the visor (D). To drain water, release the top nut of the hose at the back of the unit. Pour water into an appropriate vessel or sink.
- Regularly clean the filters placed in the chamber ground, both of them, metal and cloth (G, F). Remove the grill (Fig. 2) and clean with steam jet.

Fig. 2

IMPORTANT: Clean the filters out of the box. Working without the filters would damage the fan. When necessary, replace the used filters with new ones. Always use **MESTRA** original spare parts.

- When you notice that aspiration is less powerful, check the filter cleaning. There is a third filter near the fan. If you need replace it, first remove the back cover and then, remove the fan. You will get access to the filter. It is in a metal frame with two bolts. Replace it with the spare filter supplied with the machine. Use **MESTRA** original spare parts.

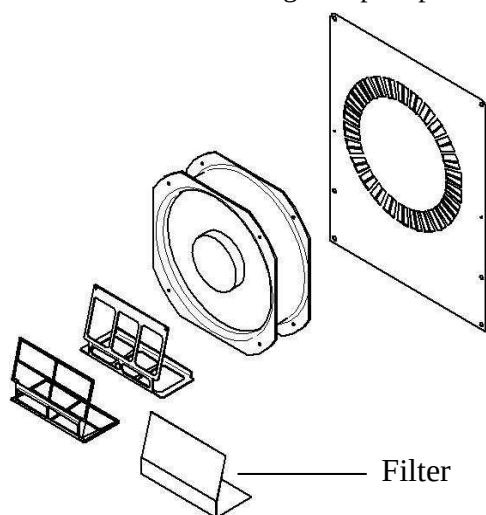
Fig. 3

- Clean the outside of the unit regularly with a cloth moistened in soapy water. Do not use solvers or other famable product.
- Use only original spare parts supplied by the manufacturer.
- To replace lamps, unscrew the holding knobs (Fig. 4).

Fig. 4

PRECAUTIONS

- Before connecting the unit to the power source, make sure the voltage is correct (grounded 230 V, 50/60 Hz).
- The cable may only be replaced by one with the same characteristics as supplied by the manufacturer.
- Attention: before using the machine, please completely read this operating manual.
- Keep this operating manual properly for future.
- Do not use the machine for a different purpose than especified in this guide.
- Do not touch the unit or the power cord with wet hands.
- Before each use, check that the water level does not reach the height of the visor (D).
- Do not fill the chamber with water, or leave an open hose inside.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Do not pull, twist or fold the power cord excessively.
- Unplug the unit before cleaning; do not wet ellectrical parts, do not use chemical products to clean it.
- Do not hold the box by the diffuser frame. Due to the weight of the machine, the diffuser may unstick. To move the box, use the handle to lift.
- Do not obstruct the ventilation grilles at the back of the unit.
- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not install the unit near to heat sources.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents,...).
- To prevent accidents, be cautious when using the unit.
- Always keep the unit off when you don't use it.
- Do not allow children or non-qualified personnel handling the unit.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not dismount, repair or modify the machine.
- If you find a defective component or a problem do not use the machine.
- Contact the supplier for questions or detailed information about problems with the unit.

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***TALLERES MESTRAITUA S.L.** is a company manufacturing tools and appliances for the dental prothesis sector, from the business address shown at the bottom of the page.*

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***I HEREBY STATE:** That the **STEAM CLEANING BOX R-080600**, with the following characteristics:*

Alto / Height:	570 mm
Ancho / Width:	520 mm
Fondo / Depth:	490 mm
Peso / Weight:	24 kg
Capacidad / Capacity:	8 l
Tensión / Power supply:	230 V, 50/60 Hz
Consumo / Consumption:	200 W

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Meets the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.

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In witness whereof, the company issues this declaration of conformity in Sondika, on 16 April 2018.

Firma / Signed:

Ignacio Mestraitua,
Director Gerente / Managing Director

MESTRA®
TALLERES MESTRAITUA S.L.
Txori-Erri Etorbidea, 60
Tel. +34 944530388 - Fax +34 944711725
mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA