R-100140

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R-100140 is a practical Ultrasonic Bath specially designed for a wide range of applications. For a long working life and an optimum performance, we recommend you to read carefully the following instructions:

INSTALLATION

- Carefully unpack the Ultrasonic Bath. Inside you will find the following accessories:
 - 1 power cord.
 - 1 plastic basket.
- Choose a flat and firm surface far from heat or vibration sources.
- Connect the machine to a grounded 230 V, 50/60 Hz power supply.
- Remove the lid, put the items in the basket. Then put them in the cleaning tank. Add water or a cleaning solution to a level between *min.* and *max.* and above the area to be cleaned. In order to prevent scratches and friction of the products, do not pile them up.
- The machine is ready for operation.

CONTROL PANEL

1. Indicator light:

Green Normal working status, ready to use.

Red Under overheat protection. 15 minutes of waiting time is needed for the light to change from red to Green. Work can resume afterwards.

2. Function button (Degas / Normal)

Press Function button.  illuminates, indicating Degas function is selected. Press On/Off button to start degas.  will flash for about 90 seconds, then the unit will return to normal cleaning function. Pressing function button before it ends will stop degassing. If additional degassing is needed after it ends, press Function button again.

Degas

When cleaning heavily soiled or greasy items, silver, copper or brass items, chemical solutions need to be added into the water. The solution may form many small droplets and take a long time to dissolve in the water. Newly added water may generate many air bubbles on the tank walls. These will reduce the cleaning effect in the beginning phase of ultrasonic cleaning. Turn on the degas function will dissipate the droplets and the air bubbles, usually in 90 seconds, and improve cleaning efficiency.

3. On/Off key:

After powering on, the LED display which is the default time and the unit is ready with normal working status.

Press ON/Off button once and the cleaning starts. the cleaning stops when the timer counts down to **00:00**. If the unit needs to be stopped before the timer runs up, press On/Off button **twice**.

4.

Degas

Illuminated: the Degas function is selected.

Flashing: the degassing is underway.

5.

Normal cleaning

Illuminated: normal cleaning is underway.

6.

Heater

Press Heater button **twice**,  flashes, indicating the heater is turned on. Press Heater button **twice** again,  dims, indicating the heater is turned off.

7. Timer button

Timer quick set button. Press timer button, LED display shows **05:00**. Pressing it again to timer it to 10:00. Each pressing makes it to increase by 5 minutes.


8. Timer button

Timer quick set button. Press timer button, LED display shows **05:00**. Each pressing makes it to increase by 5 minutes.

9. ▼ Timer decreasing button


Each pressing reduces the timer by 1 minute.

USO DEL APARATO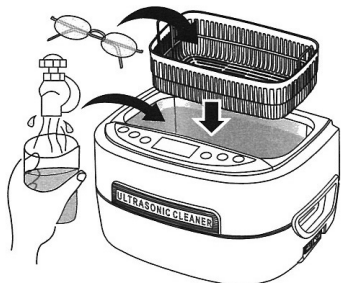

- 1 Remove the lid, put the items in the basket. Then put them in the cleaning tank. Add water or a cleaning solution to a level between *min.* and *max.* and above the area to be cleaned. In order to prevent scratches and friction of the products, do not pile them up.

If the unit is turned on without water, ultrasonic energy will not be absorbed. Once over 30 seconds, it may damage the unit or severely reduce the life of the unit.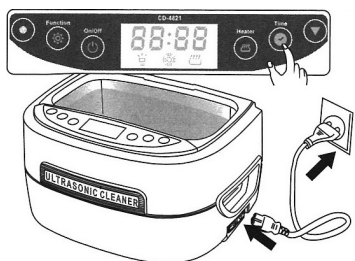


- 2 Connect the power cord (L) and turn the switch on (I). LED display shows **05:00**. This is the most commonly used time setting and working state.

If the time needs to be adjusted, press Time button and ▼ button. The time can be set between 1 to 30 minutes. Common cleaning time are 5, 10 or 15 minutes.

Prolonged cleaning time may result in:

- d. Loosening of the screws if used.
- e. Increasing the pre-existing cracks.
- f. Peeling of coating which was already separated.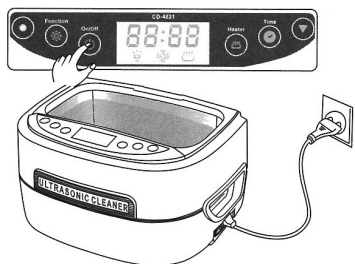


- 3 **Press On/Off button twice to start cleaning.**

During cleaning, buzzing sound can be heard from the cleaning tank, indicating the cleaning is underway. Closing the lid will reduce the noise.

LED display will count down to show the remaining cleaning time. When it displays **00:00**, the cleaning is done. To stop cleaning at any time, press On/Off button **twice**.

The unit is designed with overheating protection to avoid damage caused by extended and continuous operation. When the red light is on, the unit will not turn on even if it is forced to. The unit will idle for about 20 minutes. Then it can be restarted after the green light comes on.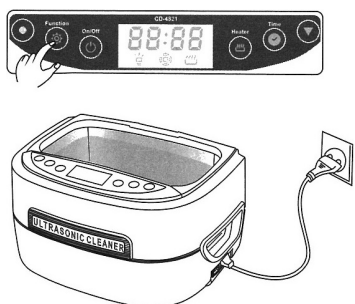

- 4 When the heater is needed, press Heater button **twice**,  flashes, indicating the heater is turned on.


When water temperature reaches 60 °C (140 °F) the heater will turn off automatically.

To ensure safety, the unit is designed with dual protections. The heater will be turned off automatically if it has been running for 45 minutes.

To speed up the mixing when solution is used, degas function can be used.


Press time button to set the timer. Press function button,  illuminates. Press On/Off **twice** to start degassing. It changes over to normal cleaning after 90 seconds.

When debris "smoke" is no longer visible, cleaning is done. If additional cleaning is needed, press time button to reset the timer and repeat the steps above.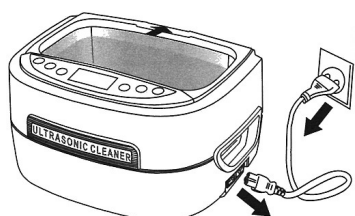

- 5 When cleaning is completed, turn off the switch, remove the lid and retrieve the items. Pour water out through the pouring recess. Clean and wipe the tank dry.


CARE AND MAINTENANCE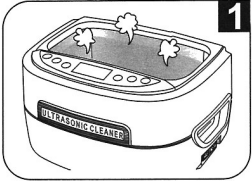

Do not turn the unit without water in the tank.


Even though the unit is designed with multiple protections, if it is turned on for over 30 seconds without water in the tank, it may damage the unit or severely reduce the life of the unit. If heating over 3 minutes, it will damage the heater.

This unit is equipped with a thermal cutoff. If the cleaning function is turned on without water in the tank, the temperature will rise continuously. The thermal cutoff will be actuated after approximately 15 minutes respectively to ensure safety. The unit will be disabled until it's repaired.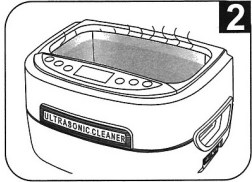


Do not turn the unit for extended time or continuously.

The unit is designed with overheat protection. If the unit has been running for 45 minutes, it is recommended to stop the unit for about 20 minutes to prolong the life of the unit.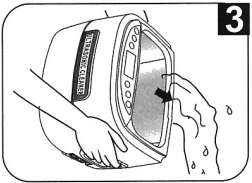


Do not keep water in the cleaning tank for a long time.

After cleaning is completed, turn off the switch then pour water out through the pouring recess. Avoid getting water inside the housing through the vents and causing damage.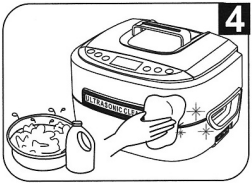

Do not spray water over the housing.

Use a towel to wipe the tank and the housing.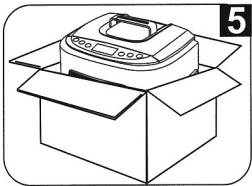

Do not expose the unit under direct sunshine for a long.

Keep the unit in a dry, cool and ventilated area.

- Dry the tank and unit with a towel. Do not wash the unit completely with water to prevent the risk of shock.
- Do not load the tank excessively; it could reduce the efficacy of the bath and damage the unit.
- Do not put very heavy or large objects in the bottom of the tank.
- Use chemical cleaning products compatible with the objects to be treated. Avoid corrosive solvents.
- When items (especially valuable items) are being cleaned for the first time, carry out a trial run first with just one item. Ask the manufacturer what cleaning solution is recommended.
- After using the unit, clean the tank and the external body with a damp cloth. Always unplug the unit before cleaning it.

PRECAUTIONS

IMPORTANT- The input voltage of the Ultrasonic Bath transducers can reach hundreds of volts. Therefore, it is very important to observe the following Safety Regulations:

- 1) Do not open the lid when the power cord is connected to the power supply.
- 2) Do not spill water or any other conducting fluid over the unit's housing.
- 3) Always connect the unit to grounded power outlets.

- ◆ Connect the unit to a grounded 230 V, 50/60 Hz power outlet.
- ◆ Do not use inflammable solvents as cleaning solutions.
- ◆ Keep the unit unplugged before filling it. Do not fill the tank above the max. mark. Do not start up the unit when the water level is below the min. mark.
- ◆ Do not allow children or unskilled people to handle the unit.
- ◆ Never submerge the unit in any liquid to clean it.
- ◆ Ultrasonic cleaning is not suitable for organic pearls.

- ◆ Do not open the unit while it is connected to the power supply.
- ◆ The machine lid may be hot after a cycle. Proceed with caution when removing it.
- ◆ If you are not sure of the effects of the cleaning solution on a given type of item, carry out a previous test with a single item.
- ◆ Do not attempt to remove any screw or panel. The user does not need to manipulate anything inside.
- ◆ To prevent electric shock, never touch the power cord or socket with wet hands.
- ◆ Never operate the unit if the plug or the power cord is damaged. If the unit does not work properly, is damaged or has fallen into the water, refer it to a technical service for examination and repair.
- ◆ Do not touch the unit if it has fallen into the water. Unplug it immediately.
- ◆ Use the unit properly according to the user's guide. Do not use accessories not recommended by the manufacturer.
- ◆ The unit should never be left unattended when it is plugged.
- ◆ Do not operate the unit next to heat sources such as radiators.
- ◆ Install the unit on a dry and flat surface.

Be careful that water does not enter through the ventilation slots. It could damage the electronic circuits.


CLEANING SOLUTIONS

Cleaning solutions can be classified into two types: those consisting of a cleaning solution mixed with water, and solvents.


The most common cleaning method is with water, as it has an acceptable capacity to dissolve inorganic substances. However, it is extremely limited in dissolving organic substances. For this type of compound, water is a good medium for the dispersion of dirt particles, and a good basis for specific detergents that dissolve organic substances.

In most cases a mixture of water and domestic washing up liquid can be used as a cleaning solution. Nevertheless, when due to the amount or nature of dirt to these cleaning solutions are not enough, specific solvents for ultrasonic baths should be used. **Under no circumstances should inflammable solvents or environmentally dangerous products be used.**


DIRT REMOVING PROCESS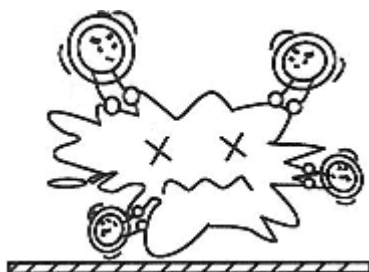

The ultrasounds generate myriads of small bubbles.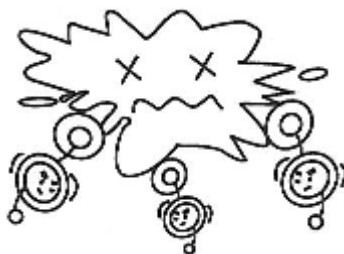

The ultrasounds make the bubbles implode constantly and impact against the object's surface

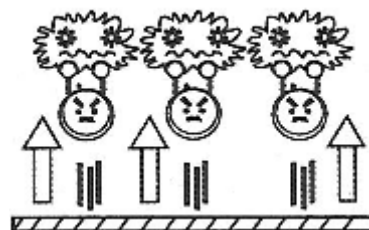The impact breaks up the dirt into many grains and separates them from the object's surface. The grains remain in the water.

TECHNICAL SPECIFICATIONS

Voltage:	230 V, 50/60 Hz
Power:	170 W
Frequency:	35 kHz
Tank material:	314 x 243 x 204 mm
Tank dimensions:	245 x 150 x 76 mm
Outside dimensions:	2500 ml <i>min: 2150 ml / max: 600 ml</i>
Max. item length:	260 mm
Tank material:	Stainless steel
Timer:	1 - 30 minutes
Weight:	2.9 kg

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TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prothesis sector, from the business address shown at the bottom of the page.

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I HEREBY STATE: That the **ULTRASONIC CLEANING BATH R-100140**, with the following characteristics:

Tensión de alimentación / <i>Voltage</i> :	230 V, 50/60 Hz
Potencia / <i>Power</i> :	170 W
Frecuencia / <i>Frequency</i> :	35 kHz
Dimensiones exteriores / <i>Outside dimensions</i> :	314 x 243 x 204 mm
Dimensiones de la cuba / <i>Tank dimensions</i> :	245 x 150 x 76 mm
Capacidad de la cuba / <i>Tank capacity</i> :	2500 ml <i>min: 2150 ml / max: 600 ml</i>
Longitud max. objeto / <i>Max. item length</i> :	260 mm
Material de la cuba / <i>Tank material</i> :	Acero inoxidable
Temporizador / <i>Timer</i> :	1 - 30 minutos
Peso / <i>Weight</i> :	2.9 kg

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Meets the low voltage requirements set out in Directive 2014/35/UE and in Directive 2014/30/UE concerning electromagnetic compatibility.

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In witness whereof, the company issues this declaration of conformity in Sondika, on 11 December 2017.

Firma / *Signed*:

Ignacio Mestraitua,
Director Gerente / *Managing Director*

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA