R-100193

E.....	MÁQUINAS DE VAPOR.....	4
GB.....	STEAM MACHINES.....	6
D.....	DAMPFSTRAHLGERÄTE.....	8
P.....	MÁQUINAS DE VAPOR.....	10
I.....	MACCHINE DA VAPORE.....	12

MESTRA®

TALLERES MESTRAITUA S.L.


Txori-Erri Etorbidea, 60

Tfno. + 34 944530388 - Fax + 34 944711725

E-mail: mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 07/09/09

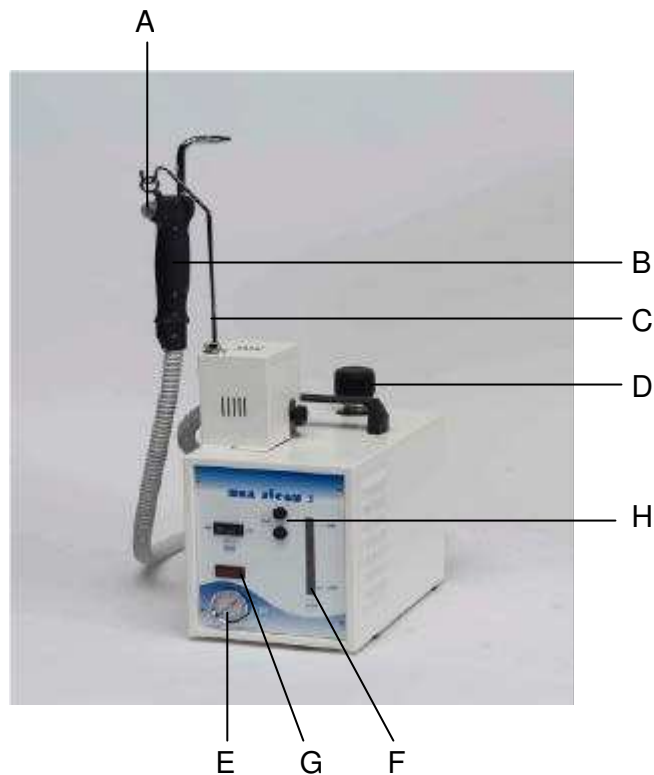R-100193

- | | |
|---|--|
| A | Botón disparo vapor / Steam trigger button / <i>Dampfpistole</i> / Botão disparo vapor / Pulsante avvio vapore |
| B | Pistola / Pistol / <i>Pistolenhalterung</i> / Pistola / Pistola |
| C | Soporte pistola / Pistol holder / <i>Füllstopfen</i> / Suporte pistola / Supporto pistola |
| D | Tapón de llenado / Filling plug / <i>Manometer</i> / Tampa de enchimento / Tappo di ricolmo |
| E | Manómetro / Pressure gauge / <i>Füllstand</i> / Manómetro / Manometro |
| F | Nivel de llenado / Filling level / <i>Hauptschalter</i> / Nivel de enchimento / Livello di ricolmo |
| G | Interruptor general / Main switch / <i>Sicherungen</i> / Interruptor geral / Interruttore generale |
| H | Fusibles / Fuses / <i>Fülltaste</i> / Fusíveis / Fusibili |
| I | Botón de llenado / Filling button / <i>Dampfpistole</i> / Botão de enchimento / Pulsante di ricolmo |

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The R-100193 is a reliable, safe and strong steam machine with a capacity of 3 litres and a pressure of 3 atmospheres. It has been designed to be used by dentists, dental technicians and jewellers. The tank is fully made of high quality stainless steel.

INSTALLATION

- Carefully unpack the unit. You will find a holding bar for the pistol and a funnel.
- Locate the unit in a flat, horizontal and stiff surface, away from vibration or heat sources. Make sure the ventilation grids of the unit are not obstructed. Keep them more than 15 cm away from the wall or any other object.
- Fix the steam pistol (B) holding bar (C) in its place.
- Before connecting the unit, make sure the voltage shown in the identification plate matches that of the power source to be used.
- Connect the unit to a grounded 230 V, 50/60 Hz power source. The source needs be correctly protected by a 6 A thermal magnetic circuit breaker (consult the installer).

FILLING THE TANK

1. When the machine is cold

- Remove the filling plug (D).
- Insert the funnel in the filling hole.
- Fill the tank with distilled water up to the mark of half-tank (approximately 1.5 litres)
- Put the filling plug (D) again and tighten it.

Always use distilled water to prevent shortening the working life of the machine. Failing to comply with this term will nullify the warranty of the machine.

2. When the machine is hot

- Turn the machine off and wait for several minutes.
- Loosen the filling plug (D) a quarter turn.
- Let the remaining pressure release.
- Remove the filling plug carefully.
- Insert the funnel in the filling hole.
- Fill the tank with, if possible, hot distilled water up to the mark of half-tank (approximately 1.5 litres). Otherwise, wait for two minutes and fill the tank with cold distilled water.

- Put the filling plug (D) again and tighten it.

OPERATION

1. Fill the tank as explained above.
2. Plug the power cord to a power source with ground connection or with a 6A thermal magnetic circuit breaker.
3. Turn the switch to the ON position
4. Wait for the "Machine ready" light to turn on.
5. Press the (A) button to activate the steam supply.
6. During the operation, check the rise of the water level on the gauge (F).

WARNING

- Do not fill the tank with a substance other than distilled water.
- Do not exceed the maximum water level.
- Aim the pistol, and therefore the steam jet, in the direction of the work you are carrying out. Do not aim the pistol anywhere else.
- The upper part of the unit can reach temperatures of more than 80 °C while in operation.
- Turn the unit off after using it.

SAFETY DEVICES

The safety devices of the machine are: plug with safety valve, pressure switch, thermostat in the heater.

When there is no water, a thermostat emits an acoustic signal and interrupts the power supply to the heater.

The unit features a fuse for the pistol (24 V) and another fuse (230 V) for the rest of the functions. In case of an operation failure, check these fuses and when needed replace them with identical fresh ones.

MAINTENANCE AND CLEANING

To ensure an optimal reliability and a long life of the unit, we strongly recommend:

- To decalcify the unit periodically.
 - Open the plug.
 - Fill the tank with water, shake and empty it.
 - Repeat this operation as many times as required.
 - Fill the tank again with distilled water.
- To clean the unit once a month by proceeding as follows:

- Disconnect the unit from the power source.
- Open the plug slowly to let the remaining steam to get out.
- Insert ½ litre of water into the tank by using the funnel provided with the unit.
- Let the water work for several minutes.
- Remove the water by overturning the tank.
- Repeat the process if required.
- Some calcium sediment could remain stuck to the steam distributor. Remove it by blowing compressed air to clear the injection pipe.
- In case of failure or breakdown of the unit, do not act directly; contact your **MESTRA** distributor.

CAUTION

- Before connecting the unit, make sure the power source is a grounded 230 V, 50/60 Hz. The source has to be protected by a thermal magnetic circuit breaker. Consult your installer.
- Do not allow children and non-qualified personnel to handle the unit.

- Open the unit lid cautiously. A sudden steam jet from the tank could cause burns.
- Locate the unit on a firm and flat surface to prevent the risk of overturning.
- Make sure the ventilation grids are not obstructed. The nearest wall or object should be at least 15 cm away.
- Disconnect the unit from the power source before proceeding to clean it.

TECHNICAL CHARACTERISTICS

Height x width x depth:	225 x 185 x 305 mm
Height x width x depth: (pistol holder included)	640 x 250 x 305 mm
Weight:	11 kg
Voltage:	220 V, 50 Hz
Power consumption:	1300 W
Capacity:	3 L
Pressure:	3 bar

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