## R-100277

E .....	MINI PULIDORA VELOCIDAD VARIABLE .....	4
GB .....	MINI POLISHER VARIABLE SPEED .....	6

**MESTRA®**

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

E-mail: [mestra@mestra.es](mailto:mestra@mestra.es) - [www.mestra.es](http://www.mestra.es)

48150 SONDIKA - BILBAO - ESPAÑA

- A Porta cepillos / **Brush holder**
- B Disco de tela / **Cloth wheel**
- C Regulador velocidad / **Speed regulator**
- D Patas de goma / **Rubber legs**
- E Tornillos sujeción portacepillos / **Brush holder bolts**
- F Cepillo / **Brush**

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The R-100277 is a simple polishing lathe especially designed to polish and grind small items in the field of dental mechanics.

## INSTALLATION

- Carefully unpack the unit. Check that it has not sustained damage during shipment. It is supplied with the following accessories:
  - 1 right-hand thread brush holder.
  - 1 left-hand thread brush holder.
  - 1 brush Ø48 (R-060440).
  - 1 brush Ø80 (R-060420).
  - 1 cloth wheel Ø110 (R-060430).
- Place the unit on a flat and firm surface remote from any source of heat or vibration.

- If you are facing the ON switch and wish to fit the brush-holder on the right-hand end of the motor shaft, select a right-hand (clockwise) thread brush-holder, so that the friction torque produced when the unit is used will help tighten the brush-holder. If you wish to fit it to the left-hand end of the shaft, select a left-hand (anti-clockwise) thread brush-holder.
- Put one brush holder each side of the polisher and fix them with the 4 bolts (E).
- Connect the unit to a 230 V, 50/60 Hz outlet with earth.

## FITTING THE BRUSH TO THE BRUSH HOLDER

- Insert the brush into the brush-holder, ensuring that the brush-holder shaft is perpendicular to the plane of the brush.
- Screw the brush onto the brush-holder as tightly as possible by hand.
- Visually check that the brush-holder shaft is perpendicular to the plane of the brush by spinning the assembly in your hand.
- Start up the unit: the spin of the motor will reveal whether the wheel or brush is off line. If there is more than 2 mm of slip, dismount and re-fit the wheel-holder or brush-holder.
- Remember that the friction torque produced during work after a brush is fitted to the brush-holder will keep it tight. This effect can be favoured by a suitable workpiece attack angle.

## CONDITIONS OF USE

For best results use the unit as follows:

- Mount an appropriate brush-holder/wheel-holder as indicated above.
- Connect the unit to a 230 V, 50/60 Hz power source.
- Select the desired spin speed (2000-7000 rpm) turning switch (C).
- Before polishing or grinding a workpiece, check the motor spin direction, as the initial workpiece attack angle depends on this. Incorrect attack angles can be extremely dangerous.
- The following protective equipment is recommended for polishing or grinding:
  - safety goggles;
  - safety gauntlets;
  - face-mask;
  - proper exhausters;
  - a protective screen suited to the characteristics and shape of the unit. TALLERES MESTRAITUA, S.L. recommends R-080200 or R-080210 screens, which are especially designed for use with the polisher.
  - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
  - Be careful with long hair.
  - No objects similar to ropes or chains that could be hooked by the machine.
- After using the unit, clean the inside of the protective screen.

## MAINTENANCE AND CLEANING

To ensure a long, trouble-free useful life for your unit, proceed as follows:

- To clean the motor and base use a cloth. Never use alcohol, solvents or other inflammable liquids.
- Never immerse the unit in any liquid.
- If you notice unusual noises or vibrations, send the unit to your technical assistance service.
- This unit is designed to work correctly on the four rubber legs (D).

## PRECAUTIONS

- ◆ Before connecting the unit to the mains, check that the power supply is 230 V, 50/60 Hz.
- ◆ Discard any cracked or damaged brush or grinding wheel: if these elements break during operation they could cause serious accidents.
- ◆ Before starting work, ensure that the brush-holder/wheel holder is firmly attached to the motor shaft.
- ◆ When handling workpieces, use safety goggles, gauntlets, a face mask and a suitable safety screen. We recommend MESTRA R-080200 or R-080210 screens.
- ◆ Do not allow children or unskilled personnel to handle the unit.
- ◆ If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.

- ◆ Never polish, brush or grind materials which may give off particles which are toxic if inhaled.
- ◆ Before starting to polish or grind a workpiece, check the motor spin direction: the attack angle depends on this direction.
- ◆ Never use the unit close to solvents, alcohol or other inflammable materials. Take all possible precautions when grinding materials which may give off sparks.
- ◆ Never use the unit without proper exhausters.
- ◆ Keep the working area clean and tidy in order to prevent accidents.
- ◆ Wear proper working cloth. Loose cloth, hanging or long hair could be caught by the machina.
- ◆ Make sure that you switch the power Off before plugging the unit, in order to avoid sudden start of the motor.
- ◆ Disconnect the unit from power supply before changing the wheel.

## TECHNICAL CHARACTERISTICS

Power supply:	230 V, 50/60 Hz
Power:	200 W
Speed:	2000-7000 rpm
Height:	135 mm
Height of shaft centre line:	95 mm
Width with brush holders:	270 mm
Weight:	3 kg